




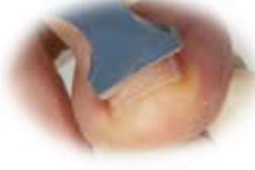




B/S Brace Magnetic Application Instructions

	<p>Preparing the Nail</p> <ul style="list-style-type: none"> • Do not soak or use any oily product on the nail before application • Reduce thick nail • Etch surface of nail to remove shine
	<p>Cleansing the Nail</p> <ul style="list-style-type: none"> • Use nail cleaner to clean surface debris from nail plate • Apply cleaner to lint free cotton swab and gently clean surface of the nail plate • Let dry thoroughly before applying brace (wipe dry)
	<p>Measuring the Nail</p> <ul style="list-style-type: none"> • Measure from side wall to side wall without overlapping: make sure the applicator does not slip • Place brace on applicator with magnetic point facing applicator • Magnet will hold the brace in place
	<p>Apply Activator</p> <ul style="list-style-type: none"> • Apply applicator to the area of nail where the brace is being applied • Leave for 20 second or until nail becomes chalky
	<p>Apply Adhesive</p> <ul style="list-style-type: none"> • Pull the brush straight out of the bottle and float a bead of adhesive across the brace • Do not paint the adhesive on the brace • Work quickly once glue is applied
	<p>Applying Brace to Nail (centre application)</p> <ul style="list-style-type: none"> • Place applicator with brace on centre of the nail • Hold in place for 5 to 7 seconds • Role to one side and hold for 5 to 7 seconds • Role to other side and hold for 5 to 7 seconds
	<p>Applying Brace to Nail (side to side application)</p> <ul style="list-style-type: none"> • Place applicator with brace on one side of nail in groove area • Hold in place for 5 to 7 seconds using medium pressure • Slowly roll to center then other side • Hold for 7 to 10 seconds
	<p>Finish</p> <ul style="list-style-type: none"> • Clean plate with cleanser • File edges and surface if tension too strong • Apply adhesive, let dry and polish if desired