

SILVERFOX

F-314B

Beauty Instrument

User Manual

Welcome to use our products. A0
Please read the user manual carefully before using the products.
Our company reserves the right to improve design and modify
the products without prior notice.



About this manual	1
Safety Guidance	1
F-314B High-Frequency's Introduction & Parameters	2/3
Troubleshooting	4

About this manual

This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, SILVERFOX will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of our company SILVERFOX.

Safety Guidance


**To ensure your correct and safe use of this product, please read all the instructions before use!
Please follow all safety precautions!**

	Warning!	<p>Indicates a potential dangerous situation that, if not avoided, could result in serious loss of life.</p> <ul style="list-style-type: none">• This product is only for beauty practitioners with professional qualifications.• Do not use or store the product in a damp place such as bathroom to avoid malfunction and danger.• Do not use this product near flammable items (e.g., alcohol, thinner, gasoline, nail polish remover, sprays, etc.).• Do not use this product under the following conditions: when the power cord or plug is damaged or gets hot; when the plug in the power socket comes loose.• Do not use wet hands to plug and unplug power cord connectors and use this product.• Please make sure that the working voltage of this product conforms to its rated voltage.• Before use, make sure that the power plug and the power cord connector are fully plugged in, so as not to cause danger.• When not using this product, please turn off the power and pull out the plug.• Please regularly clean the dust accumulated on the power plug and power cord connector to avoid failure of insulation caused by moisture, dust and other factors.• Be sure to use the special power cord assembly for this product. Do not use the power cord assembly on any other appliance.• Do not wrap the power cord around the product, otherwise it may distort and malfunction the power cord.• Keep out of reach of children. Do not allow children to use this product.• Please do not modify, disassemble or repair this product by yourself. If you need to repair it, please contact the manufacturer, agent or professional maintenance personnel to check and repair it, so as not to cause danger.• Please contact distributor, service center or related personal with appropriate skill and experience for repairing if the plug is damaged, in case of any danger.• Please be sure to use this product according to the purpose specified in this manual. Do not use accessories not recommended by the manufacturer.
	Attention!	<p>Represents a potentially dangerous situation which, if not avoided, can result in general injury to person.</p> <ul style="list-style-type: none">• Operators should pay special attention to the safety of personnel and equipment when using.• If the product is abnormal, please stop using it immediately and contact the manufacturer or dealer for relevant treatment.• This product and related accessories should be handled lightly.• Please properly handle the packaging bag of this product to avoid accidents such as asphyxiation of infants, and keep it out of reach of children.

◆ F-314B High-Frequency (Only used for beauty)



1. Power Switch
2. Power Energy Increase +
3. Power Energy Decrease -
4. Time Increase +
5. Time Decrease -
6. Start/ Pause

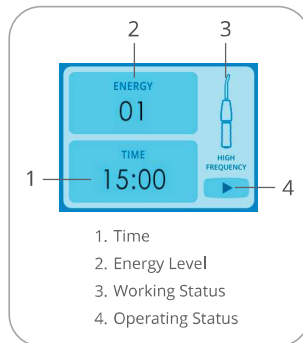

Operation Method

1. Insert the High-Frequency into the corresponding socket of the Equipment. When the equipment is connected to the power cable properly, the "Power switch" is displayed in red. Press the "Power Switch" to enter into the working mode and it would show in white in the screen. The equipment defaults to the continuous wave output state; thus the energy output of level 1 is set to 15 minutes.
2. There are various modes can be selected based on your need, choose the appropriate electrotherapy tube to install on the handle of high frequency electrotherapy.
3. Press the "Time increase/decrease button" which allows you to set the Working time from 1-30 minutes.
4. Press the "Start"/"Pause" button to start the operation. The power energy level is adjustable which allows being set from level 1 to 10.
5. When you want the Ultrasonic probe to stop working during the operation you can press the "Pause" button.
6. Press the "power switch" to turn off the equipment when the treatment is end, and the "power switch" will display red at this time. Clean the electrotherapy tube with 75% alcohol after the unplugging the power plug, and store in a dry place.

Recommended using time

- 1) Oily skin: 10 to 15 minutes
- 2) Dry and dehydrated skin: 3 to 5 minutes

Warning: Do not put the metal part of the electrotherapy tube in contact with water

Operation Cautions


F-314B

⚠ Warning

- It is forbidden to touch the output interface of the main machine of the high-frequency electrotherapy beauty instrument or the output interface of the handle of the high-frequency electrotherapy when the main machine is powered on.
- Do not point the tube directly or in contact with the eyes area.
- This product is prohibited for the following people and parts:
 - ① Pregnant women and those equipped with adjusters (e.g. cardiac adjusters).
 - ② People with vascular diseases and high skin sensitivity.
 - ③ Having metal implants or metal dentures embedded in the body.
 - ④ People with skin wounds and other conditions.
 - ⑤ Body parts with thick hair.
- Do not use skin lotions containing alcohol, as alcohol is highly flammable. If alcohol-based toner is required, only apply it after using this product.
- Do not grasp the junction between the high-frequency handle and the electrode tube by hand during operation to avoid being shocked.
- The skin retention time of each part should not exceed 5 seconds.
- This function shall not be used for other than cosmetic purposes.

⚠ Attention

- Before use, check whether the electrode tube or the metal head on the tube is loose and whether the electrode tube cracks or is damaged. If so, do not use it and replace the electrode tube immediately.
- Reduce the current intensity to zero gradually, turn off the power switch, and pull out the power cord plug before inserting or pulling out the electrotherapy tube.
- When inserting or pulling out the electrode tube, first wear rubber gloves and hold the electrode tube with thumb and forefinger: insert the tube to the bottom of the hole until it is firm, and do not push the tube with the palm to prevent the glass from breaking and causing injuries; It is forbidden to pull out vigorously, so as to prevent the metal head on the electrode tube from falling off or breaking, which may cause injury.

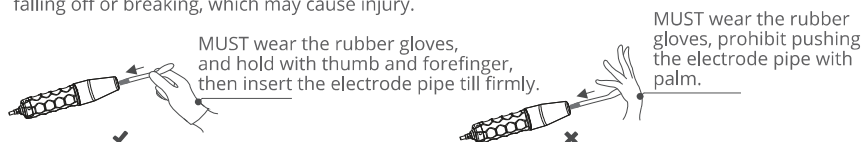

- When in use, the high voltage current output from the electrotherapy tube will cause interference.
- It is forbidden to approach the host machine and other electrical appliances of the high-frequency electrotherapy beauty instrument. If it interferes with the host, it may cause the host to not work or crack.
- When the electrode tube is less than 2.5cm from the skin but not in contact with the skin, the current intensity should be reduced immediately if sparks emits to irritate the skin. If the customer still feels the spark after the above operation, a small piece of dry cotton towel (nylon or other synthetic fiber towels are not allowed) put on the customer's face.
- When using, always keep the electrode tube on the customer's skin. High-frequency indirect care: when turning on and off the instrument, one hand must always be in contact with the customer's skin, never leave the customer's skin. Both sides feel the current and it causes discomfort.
- Turn off the power before pull out the power cord plug after use, and pull out the electrotherapy tube from the high-frequency electrotherapy handle, then sterilize and keep it dry before putting it into the accessory box.

Product Parameters

Product model	F-314B
Power	AC 100V-120V or AC 220V-240V, 50/60Hz
Power Rating	15W
Product size	20.8×23×7.5 cm
Package size	37×27.5×14 cm
Package contents	Main Unit × 1
	Accessories Shelf × 1
	High frequency electrotherapy handle × 1
	Spoon shaped electrotherapy tube × 1
	Dot shaped electrotherapy tube × 1
	Rectangular electrotherapy tube × 1
	Mushroom shaped electrotherapy tube × 1
	Power cord × 1

Troubleshooting

The Fault Phenomenon	Reason	Solution
The main machine does not work or crashes.	1. No power.	1. Check whether the fuse in the fuse bin in the power base is burned out. If burned out, replace the corresponding specifications of the new fuse. 2. Press the start button, the button will light up after power.
	2. The power cord is not properly inserted or connected.	2. Plug or connect the power cord properly.
Display screen doesn't work.	1. No power.	1. Turn on the power.
The output of high frequency electrotherapy is weak.	1. The tap position is not set well; 2. Aging after long-term use of electric therapy rod.	1. Adjust the high-grade position; 2. Replace the electrotherapy accessories.

If problem is still not solved after above operations, please contact maintenance center or supplier to solve it.



Do not dispose of old or defective appliances in domestic garbage;
this should only be done through public collection points.

- ▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.