

SILVERFOX

F-319B

Beauty Instrument

User Manual

Welcome to use our products. A0
Please read the user manual carefully before using the products.
Our company reserves the right to improve design and modify
the products without prior notice.

About this manual ----- 1

Safety Guidance ----- 1

F-319B Diamond Dermabrasion's Introduction & Parameters ----- 2/3



Troubleshooting ----- 4

About this manual


This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, SILVERFOX will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of our company SILVERFOX.

Safety Guidance


**To ensure your correct and safe use of this product, please read all the instructions before use!
Please follow all safety precautions!**

	Warning!	<p>Indicates a potential dangerous situation that, if not avoided, could result in serious loss of life.</p> <ul style="list-style-type: none">• This product is only for beauty practitioners with professional qualifications.• Do not use or store the product in a damp place such as bathroom to avoid malfunction and danger.• Do not use this product near flammable items (e.g., alcohol, thinner, gasoline, nail polish remover, sprays, etc.).• Do not use this product under the following conditions: when the power cord or plug is damaged or gets hot; when the plug in the power socket comes loose.• Do not use wet hands to plug and unplug power cord connectors and use this product.• Please make sure that the working voltage of this product conforms to its rated voltage.• Before use, make sure that the power plug and the power cord connector are fully plugged in, so as not to cause danger.• When not using this product, please turn off the power and pull out the plug.• Please regularly clean the dust accumulated on the power plug and power cord connector to avoid failure of insulation caused by moisture, dust and other factors.• Be sure to use the special power cord assembly for this product. Do not use the power cord assembly on any other appliance.• Do not wrap the power cord around the product, otherwise it may distort and malfunction the power cord.• Keep out of reach of children. Do not allow children to use this product.• Please do not modify, disassemble or repair this product by yourself. If you need to repair it, please contact the manufacturer, agent or professional maintenance personnel to check and repair it, so as not to cause danger.• Please contact distributor, service center or related personal with appropriate skill and experience for repairing if the plug is damaged, in case of any danger.• Please be sure to use this product according to the purpose specified in this manual. Do not use accessories not recommended by the manufacturer.
	Attention!	<p>Represents a potentially dangerous situation which, if not avoided, can result in general injury to person.</p> <ul style="list-style-type: none">• Operators should pay special attention to the safety of personnel and equipment when using.• If the product is abnormal, please stop using it immediately and contact the manufacturer or dealer for relevant treatment.• This product and related accessories should be handled lightly.• Please properly handle the packaging bag of this product to avoid accidents such as asphyxiation of infants, and keep it out of reach of children.

◆ F-319B Diamond Deamabrasion

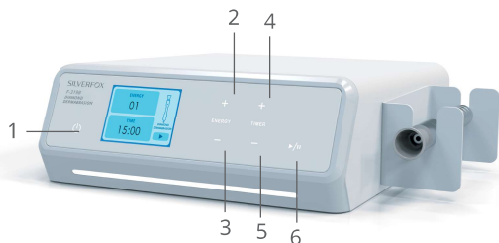

1. Power Switch
2. Power Energy Increase +
3. Power Energy Decrease -
4. Time Increase +
5. Time Decrease -
6. Start/ Pause

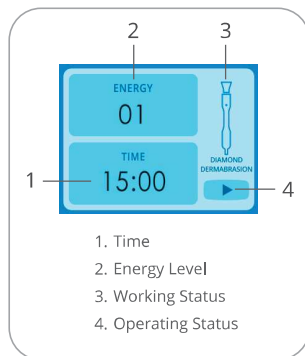

Operation Method

1. Insert the Suction pen or Stick into the host socket. When the equipment is connected to the power cable properly, the "Power switch" is displayed in red. Press the "Power Switch" to enter into the working mode and it would show in white on the screen. The equipment defaults setting is level 1 for 15 minutes.
2. Select the appropriate microcrystalline grinding head for the site and effect of care.
3. Press the "Time increase/decrease button" which allows you to set the working time from 1-30 minutes.
4. Press the "Start"/"Pause " Suction Stick start the operation. The power energy level is adjustable which allows being set from level 1 to 10 by "Energy increase/decrease button"
5. When you want the Suction Stick to stop working during the operation you can press the "Pause" button.
6. Press the "power switch" to turn off the equipment when the treatment ends, and the "power switch" will display red at this time. Clean the Suction Stick and microcrystalline grinding head with 75% alcohol after unplugging the power plug, and store it in a dry place.

**Warning**

- This function is not applicable to patients with hypertension, heart disease and diabetes.
- Pregnant women, lactation women, menstrual women should not be used.
- It is prohibited for use by patients with infectious diseases and cancer.
- Botox is banned for people who have received injections within a year.
- People with scar tissue and collagen deficiency should not use this function, otherwise it will cause permanent skin damage.
- This function shall not be used for other than cosmetic purposes.
- Prevent children from being exposed to the product and treating it as a toy, thereby causing danger.

**Attention**

- Analyze the user's skin condition before care, and then select the right grinding head and proper suction.
- After each use, the filter at the suction stick needs to be cleaned or replaced
- After care, clean the face with water.
- After the nursing operation, the grinding head should be disinfected with 75% alcohol and stored properly and dry.

Product Parameters

Product model	F-319B
Power	AC 100V-240V, 50/60Hz
Power Rating	30W
Product size	20.8×23×7.5 cm
Package size	37×27.5×14 cm
Package contents	Main Unit × 1
	Accessories Shelf × 1
	Micro-carving accessories (including 9 kinds of micro-grinding heads) × 1
	Power cord × 1

Troubleshooting

The Fault Phenomenon	Reason	Solution
The main machine does not work or crashes.	1. No power.	1. Check whether the fuse in the fuse bin in the power base is burned out. If burn out, replace the corresponding specifications of the new fuse. 2. Press the start button, the button will light up after power.
	2. The power cord is not properly inserted or connected.	2. Plug or connect the power cord properly.
Display screen doesn't work.	1. No power.	1. Turn on the power.
Accessory probe doesn't work.	1. Poor interface contact.	1. Unplug the output cable, and then reinsert the socket.
Weak output for Cold and hot hammer, ultrasonic, scrubber, RF, Galvanic and other accessories.	1. The tap position is not set well; 2. The high temperature of the accessories leads to low output power.	1. Adjust the high-grade position; 2. Appropriate use, do not open for a long time. Turn off the power switch when accessories are not in use.
Dermabrasion has weak suction.	1. The tap position is not set well; 2. The inside of the Dermabrasion blocked; 3. Aging of air tube; 4. The interface between the air tube and the suction pen is loose.	1. Adjust the gear position and adjust the air regulating valve; 2. Replace the Dermabrasion accessories; 3. Replace the new air tube and insert the interface.
Sprayer with weak spraying effect.	1. The tap position is not set well; 2. The water refill instrument is blocked inside; 3. Aging of air tube; 4. The interface between the air tube and the water refill instrument is loose; 5. The water in the bottle is insufficient.	1. Adjust the high-grade position; 2. Replace the accessories of the water refill instrument; 3. Replace the new air tube and insert the interface; 4. Pay attention to the content of water in the water bottle and add water in time.
The suction of the pen is weak.	1. The tap position is not set well; 2. The pen suction tube is blocked inside; 3. Aging of air tube; 4. The interface between the air tube and the suction pen is loose.	1. Adjust the high-grade position; 2. Replace the suction pen attachment; 3. Replace the new air tube and insert the interface.
The output of high frequency electrotherapy is weak.	1. The tap position is not set well; 2. Aging after long-term use of electric therapy rod.	1. Adjust the high-grade position; 2. Replace the electrotherapy accessories.
The host jitter.	1. Accessories are blocked.	1. Check whether the air tube of relevant accessories is blocked.

If problem is still not solved after above operations, please contact maintenance center or supplier to solve it.



Do not dispose of old or defective appliances in domestic garbage;
this should only be done through public collection points.

- ▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.