

3021

12 in 1 Beauty Instrument

Function:

Skin Scrubber, Ultrasonic, Galvanic, High-Frequency, Rotary Brush, Vacuum, Spray, Magic Glove, Cold & Hot Hammer, Magni-fying Lamp, Hottowel cabinet, Steamer.



**PROFESSIONAL
BEAUTY INSTRUMENT**

Manual of 3021 Combination

【Features】

“3021 combination ” is a free combination which has 12 different beauty funtions as the free options .

The 12 optional functions include :Facial steamer, Hot towel cabinet, Magnifying lamp, Skin scrubber, Ultrasonic, Galvanic, High-frequency,

Rotaray Brush,Vacuum,Srpay, Magic glove, Hot&cold hammer.

Base on the orgnizer trolley- 3020, you could put inside each different single function to compose a different free combination to meet your demands.

It is really smart and saving space. It will be the right choice for you.

Check after receiving :

When you receive it, Please check if the carton is in well condition.

The full combination of 3021 including 4 cartons , the packing details as below

Mark	Carton Size	Contents
3021-1	82.5*53*47	Trolley+ accessories
3021-2	90*33*32	F-300DTT+F-311A~F-316A
3021-3	82*43*13	1005T
3021-4	37.5*31*28	T-01




Caution

The carton “3021-1” is fragile, please don’t position this carton upside-down or drop it randomly . please carry it in care.



【The Sketch map of the accessories】

The below is the sketch map of the accessories , if you find there is any missing or broken,please contact your nearest distributor or contact with us directly.

Carton 1: (3021-1)

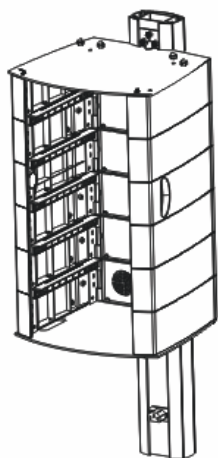


The main frame

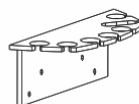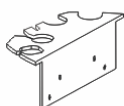

1.The accessory holder (left)

2.The accessory holder (right)

3.The glass board of the trolley

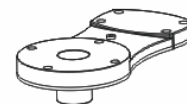

1.The motherboard of the trolley

2.The decorative cover of the wheels

3.The fixing board of the steamer






















1.Power lead












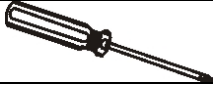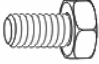

2.Wheel




3.The mag.lamp holder

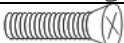
Sep. No.	Item code	Name	Picture	Specification	Qty(pc s)
1.	F-311A	The cover of scrubber			1
2.	F-311A	Skin scrubber			1
3.	F-312A	The ultrasonic probe (eye)			1
4.	F-312A	The ultrasonic probe (face)			1
5.	F-313A	Glvanice roller proble (pair)			1
6.	F-313A	Glvanic ball probe (pair)			1
7.	F-313A	The connecting line of Galvanice			1
8.	F-314A	High-frequency electrodes (set)			1
9.	F-314A	High-frequency handle			1
10	F-315A	Brush handle			1
11	F-315A	Brush head(set)			1
12	F-315A	Brush connecting line			1
13	F-316A	Spray bottle			1
14	F-316A	Vacuum handle			1
15	F-316A	The vacuum head			1
16	F-316A	The windpipe			1
17	3020	The fixing tool 1			1
18	3020	The fixing tool 2			1
19	3020	The fixing tool 3			1

20	3020	The fixing tool 4			1
21	3020	Screw		M8*5	4
22	3020	The spring mat		Φ8	4
23	3020	Gasket		Φ8	4
24	3020	Screw		M4*8	4
25	3020	The fixing element 1		M3*10	12
26	3020	The fixing element 2		M3*16	2
27	3020	The fixing element 3		M8*16	8
28	3020	The fixing element 4		M3*16	6
29	3020	Screw cover		M6	4
30	3020	PVC flat mat		Φ8	3


Carton 2: (3021-2)

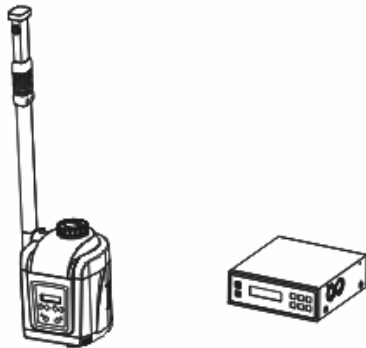


Carton 3: (3021-3)

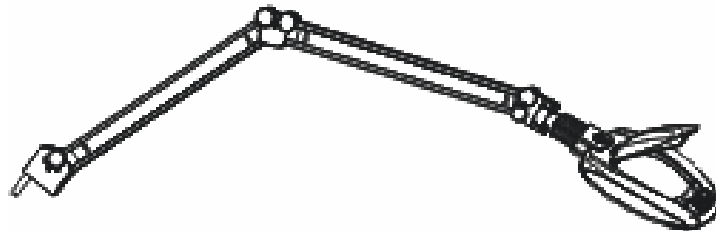


Carton 4: (3021-4)

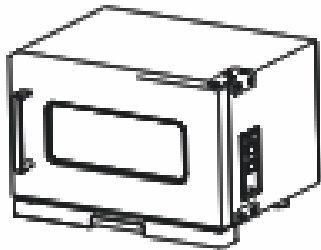


【The installation manual of trolley organizer】
The installation of wheels and the motherboard:

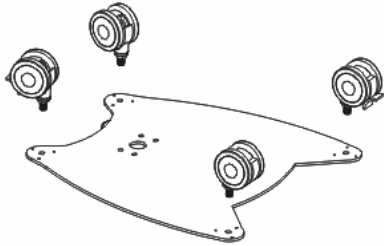


Pic.①

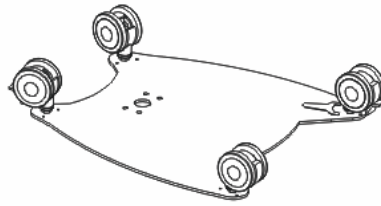


Pic.②

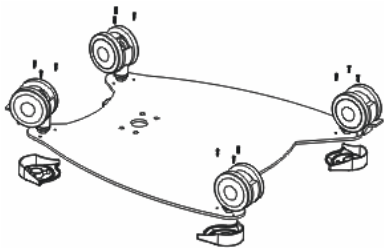



Pic.③

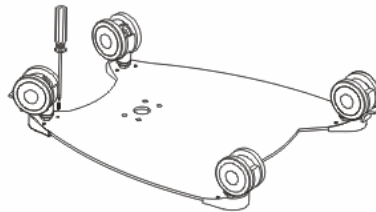



Pic.④

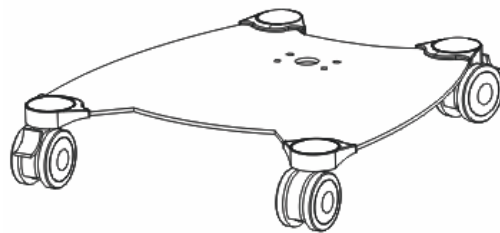


Pic.⑤

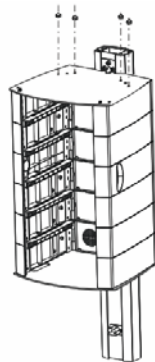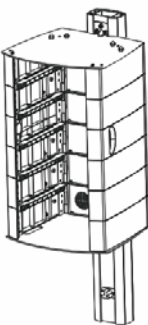


Pic.⑥

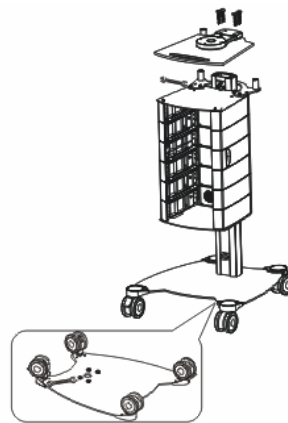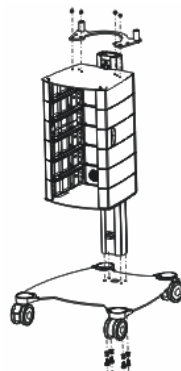



Pic.⑦

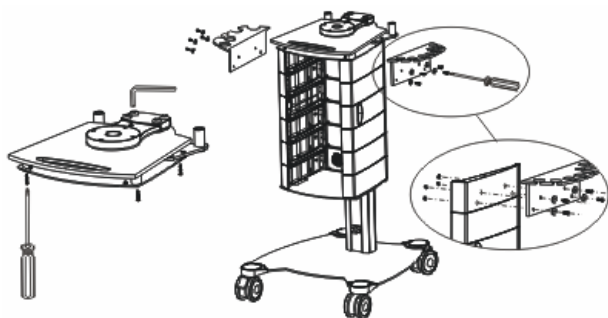

Pic.⑧

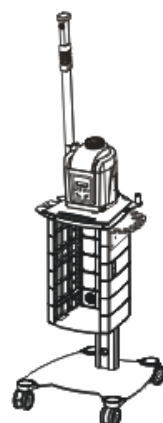

Pic.⑨

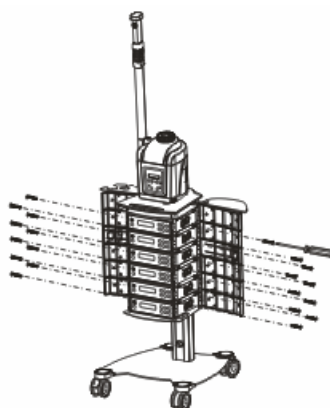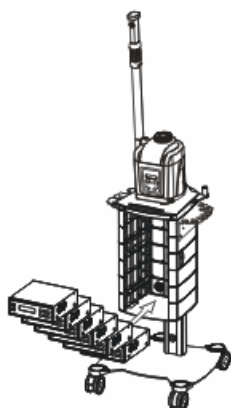

Pic.⑩

F-311A
F-312A
F-313A
F-314A
F-315A
F-316A



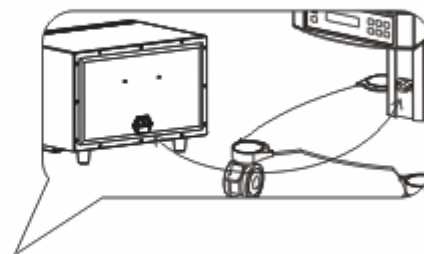


图 ⑪

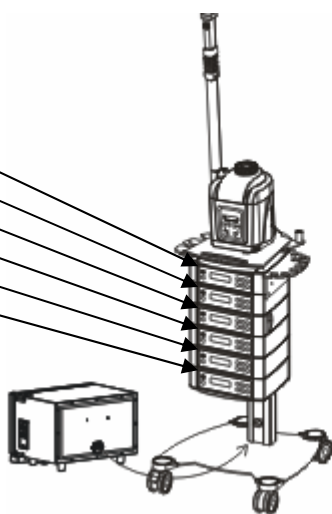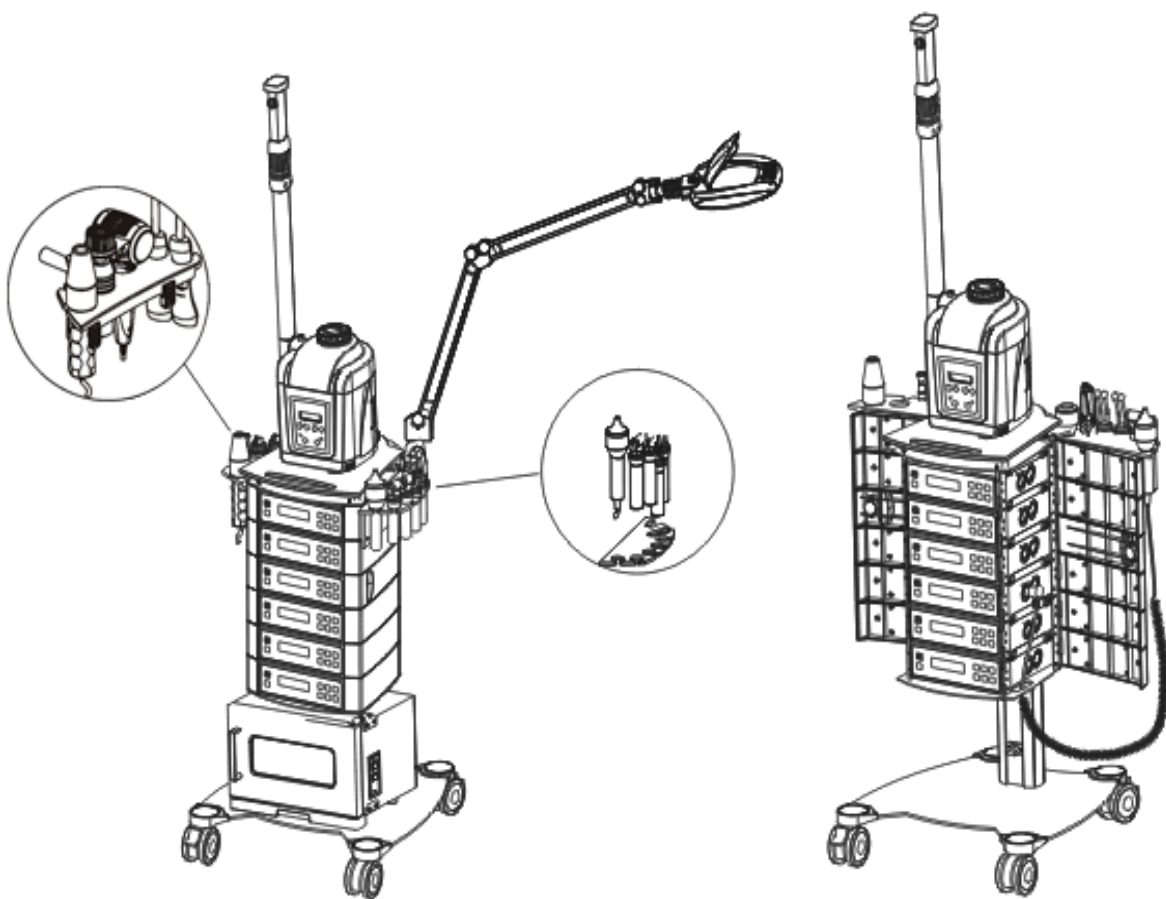

F-300DT Facial steamer

【F-300DT digital facial steamer】

We applied the latest new concept in designing this new facial steamer-- F-300DT series.

Its appearance, structure are totally different from the traditional facial steamers.

We applied the LCD display screen for this new steamer, The customer can see its meantime process of the working status easily..

The new heating system was applied for this steamer which help it effectively shorten the water boiling time by compare with the traditional heating system. On another hand, the inner space of the cup has been fully used, the cup is easy cleaning.

The cup is made by the anti-head material, which can not only guarantee its working stability in the high-tem condition, also can help reduced the surface's temperature to avoid the scald happens.

It is also equipped with time pre-set, three level spray adjustment, and an automatic water level detector, it is set maximum and minimum water level alarming system, prevent water spraying upon over maximum water level and heater damaged low over minimum water level, and prevent heater caused damage for empty container working by the user for their carelessness.


This steamer is designed as the part of the combination, could be used in the free combination 3021, also with F-300DT, and F-300E.


【Functions】

1. Intenerate the necrotic skin cells for cleaning in the subsequent treatment.
2. Open up the skin pore for deep cleaning inside.
3. Vapor penetrates into pores to soften and clean the grease, black head, make-up remnants and dirt.
4. Vapor may help skin pores release toxin.
5. Vapor may soften wrinkle for the time being.
6. Activate the blood circulation, moisten the skin, improve metabolism and renew the skin.
7. Help to treat acne, stains, whiten and nourish skin by using appropriate herbs.

【Introduction of panel】

- ① The display of the Ozone
- ② The display of the Vapor
- ③ The display of the energy
- ④ The display of the timer
- ⑤ "+" for Energy
- ⑥ "-" for Timer
- ⑦ "+" for Timer
- ⑧ "-" for Energy
- ⑨ The light of Ozone
- ⑩ The light of Vapor
- ⑪ ON/OFF of Ozone
- ⑫ ON/OFF of Vapor

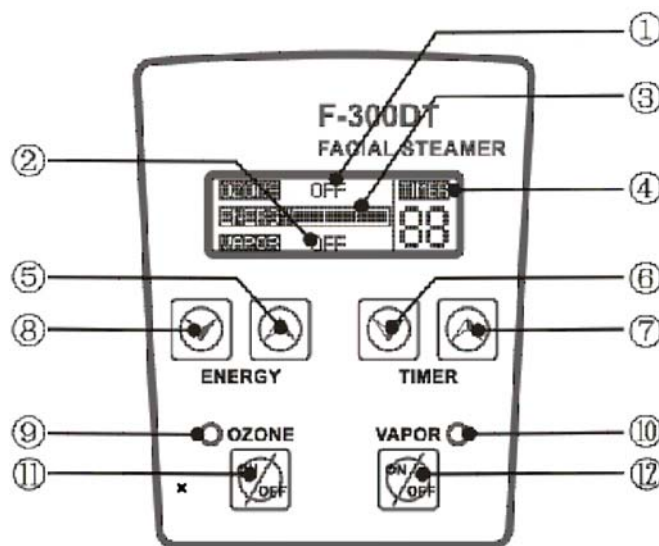


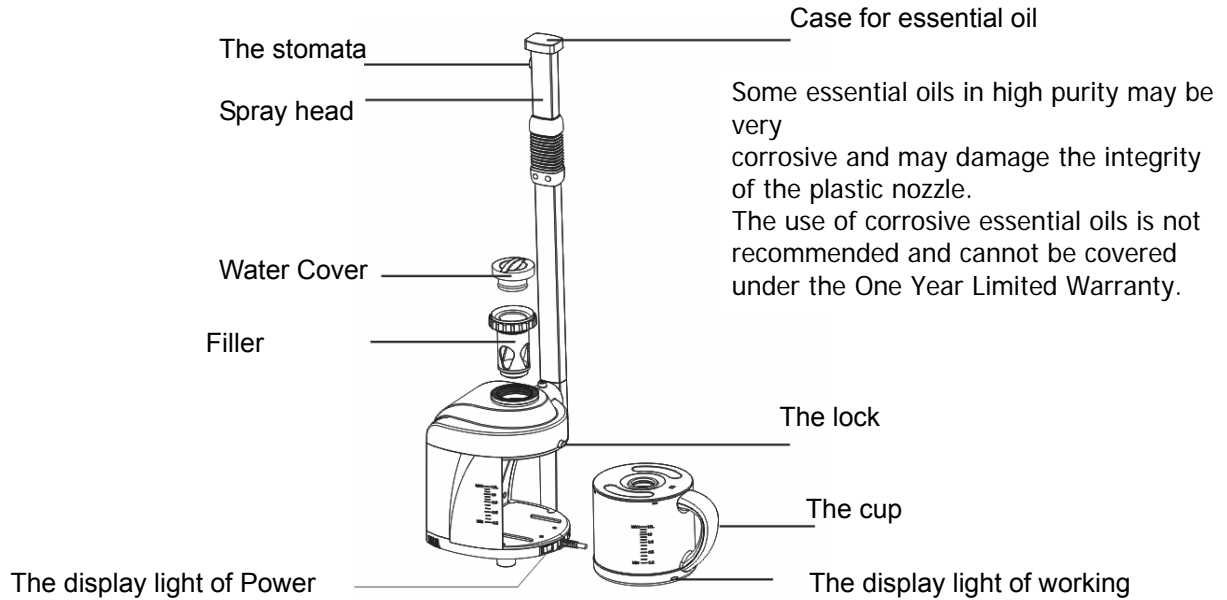

【The manual of operation】


1. when you want to take out the cup ,Turn off the filler firstly (pic. ①), press the grey small button (see pic. ②),then you can take the cup out . You can also open the cover to add the water like the pic. (3) shows.

Please be sure that the water inside of the cup should be not exceed the line"MAX" or lower the "MIN". Please screw the cover tight otherwise the "unlock" will show on the screen when turn on the vapor.

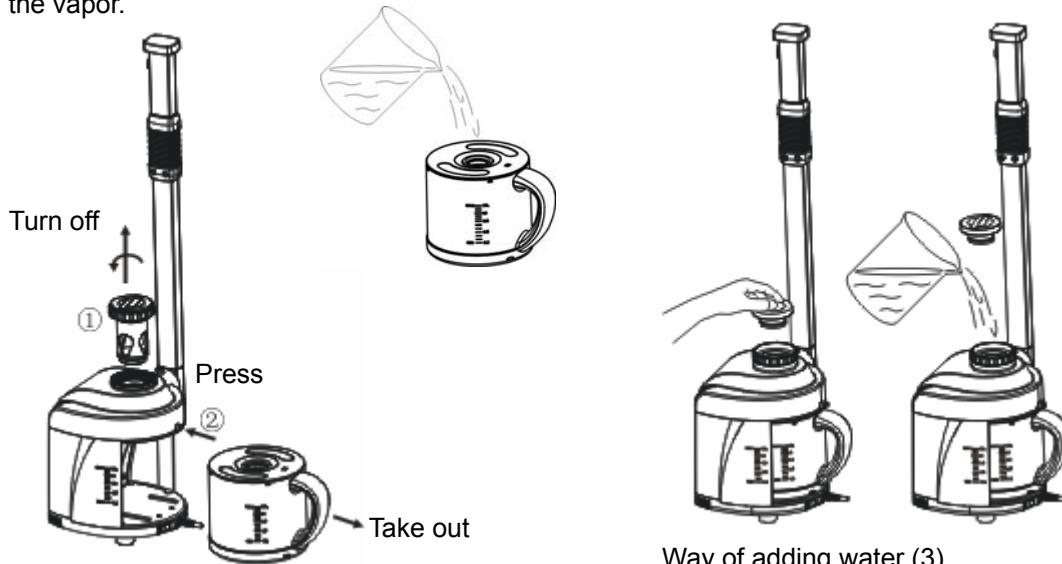

2. Connect the power supply and connect the power lead on the plug which positioned on the back of the aluminum bar (see picture on the right side)


3. The steamer will self-check after electrify ,then turn to standby status.

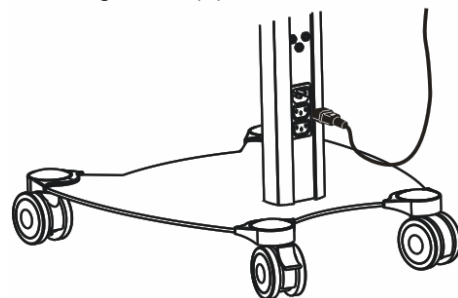


4. Pre-set the time and energy (you are suggested to turn the energy to the highest level ,then water will be boiled quickly).
5. Press “ON/OFF OZONE” to turn on the Ozone while the water start boiling , the display light turn on , the steamer start to spray out and with the smell of ozone. Now the vapor is on the normal spraying status and could be applied for the facial beauty operation.
6. if the aromatic spray is requested, you could open the aromatic case and drop into 1 or 2 drops essential oil and close the cover.

【Caution】

1. This steamer has self-check function, if the water is lower the “MIN” line, then the steamer will be instructed not working and meantime emit the sound “Do”Do”..to remind you to solve the mentioned problem before start working.

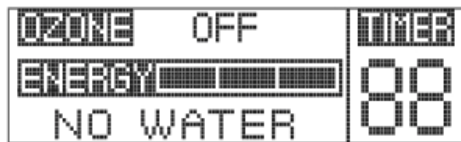

No water

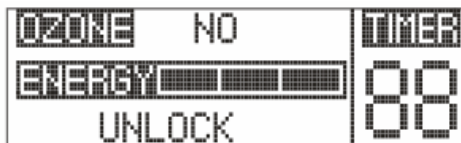


Leaking ,The cover is not fixed well

2. During operation, The vapor should be stably and equably spray out before applying on the face. The spray stomata should keep the certain distance with the customer. See the below attached reference sheet.
3. Please keep this facial steamer away from the high-interferential device (like: High-frequency instrument). If the circuit board or LCD sreen was incautionly interferenced , the steamer may stop working suddenly or the quality of performance will be lowered. Normally , please turn off the power ,then re-start after 15 seconds. The steamer can recovered to the normal working status. If it stilled not been fully recovered , it is probably because the interference is too strong and cause the inner parts damage or broken, if so, please turn off the power and contact with the local distributor for maintaince or replacement.
4. if the herb cover has not been screwed tightly enough which may cause the vapor leakage, but please also don't srew the cover over-strengthed , otherwise it may shorten the lifespan of the sealing ring of the vapor dispensor . to avoid any harmful from any means of the leakage, you are suggested to change the sealring every three months. For each steamer, we have put in one extra sealring for the replacement .
5. please don't take out the herbal case to pour the water , the water drops from the herbal case may cuase the short-circuit.




Don't take out the herbal case to pour the water

6. The cup could be only pull out after taking away herbal case, Please don't pull out the cups when the herbal case still in the steamer it may cause damage (see picture below). Please be noted : the cup should be pull out everyday after use, please wash off the bead or steams when have , keep the cup dry. When wash-clean the cup , please be aware of the plug has been away from the power supply, otherwise, any drops of the water may cause the short-circuit and damage the inner elements . meanwhile it may dangerous to people.

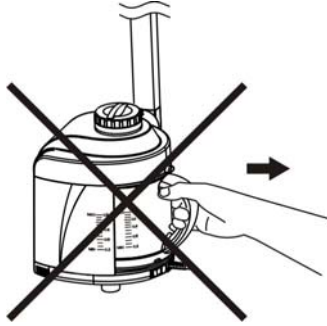

Don't pull out the cup by force when the herbal case still in

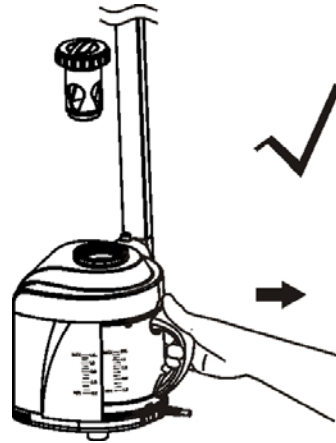




For difference skin ,please keep away from the difference distance

The below parameter just for your reference :

Skin type	Distance	Time
The oily skin 、 The blood circulation is not good	35-40cm	Around 15 minutes
The normal and healthy skin	35-40cm	10 minutes
The sensitive and dry skin	40-45cm	5 minutes
The dilatation of capillary vessel or inflammation	40cm	5 Minutes

F-311A Skin scrubber

【F-311A Introduction】

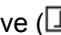1. By means of ultrasonic waves, the instrument is helpful in clearing blackheads, pimples, dirty, cosmetic remains, grease and acne of skin.
2. By means of GALVANIC electrons movement, it is magical for removing wrinkle, alleviating dark spots.
3. There is three optional waves : the sine wave (), Mix wave () and square wave ()
4. LCD screen is applied for this device.

【The operation manual of scrubber】

A: Make well connection with power line, and scrubbing shovel.


B: Turn on general power switch at rear panel. Turn on power switch of microcircuit“POWER”At this moment, the main unit is in the pending status.

C: Choose the output waveform, energy and time (The default settings of booting is continuous-wave, energy for the greatest ,15 minutes), You can also change the default settings when in the course of output

D: Press START / PAUSE, In this state, hold the shovel blade to the thumb and forefinger, respectively, hold down the shovel blade on both sides of the detection switch (as shown) on the Blade started. Work Blade on the screen associated with movement patterns, to suspend the work of Blade, Blade will be a static display pattern. As long as the cosmetic operation to shovel head 45 ° inclined to reach out to the customers' site of beauty and gently slow moving forward shovel, then it can take operation of cleaning skin.

【The introduction about the panel】

- ①Power Switch
- ②Shovel Working State Display
- ③Wave Display
- ④Timer
- ⑤ENERGY“+”button
- ⑥TIMER“+”button
- ⑦ON/OFF
- ⑧ENERGY
- ⑨Switch bottom of waveform
- ⑩ENERGY“-”button
- ⑪TIMER“-”button

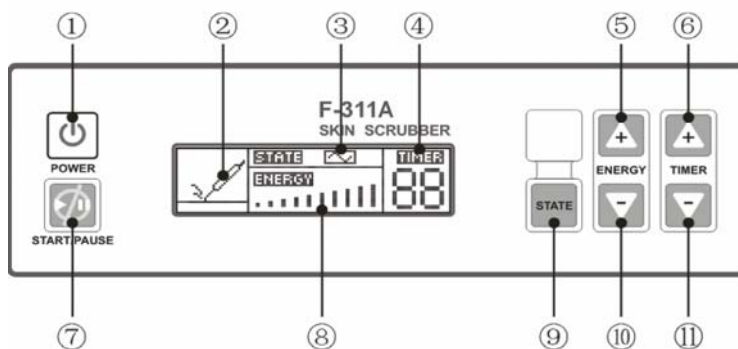

【Attention】

- 1、 The host boot time has been interval of 0.5 seconds for successive call, Displaying panel showing the shovel skin like the knife icon has been flashing, this phenomenon could not be detected for the host skin shovel knife, maybe it's resulted by the shovel knife's accessory connecting line's poor contact with the host , check whether the poor contact with the host annex. Rule out the poor contact in order to restart the computer after normal use
- 2、 If the check does not have an adverse phenomenon of contact, skin scraper blade may be damaged or host annex no output, professional maintenance personnel need to ask to check the repair. Blade check out if there is fault for the replacement of the annex, use the original production

of the parts to replace.

3. In this operation, the skin should be kept moist. For use in cleaning liquid nutrient solution added should be advised to select a non-viscous or the use of diluted liquid supplement (Prohibition of the Use of oil essence or skin care products).

4. Before the use of pre-set up as far as possible to use wave energy, time, to avoid the process of cleaning fluid in the other supplementary viscous liquid splashing in the host.

【Warning】

1. With the following diseases, not suitable for the use of this instrument, such as heart disease, infectious disease patients, allergic patients and the patients who have metal in body during therapy period.

2. Do not allow a probe in output status for too long without beauty operation or it will damage the probe. Prohibit the use of head surface deformation, defect or polished Blade-off so as not to scratch the skin

3. When the lotion of the skin surface cannot moisten skin, it should be added in time

4. Please mind the connecting line between main unit and accessories, do not pull and pick too much.

5. When operation, avoid water and other corrosive liquid splashed onto main unit.

6. If it is unusual of the instrument, please contact local sales agent and ask for repair by professional repairman. Prohibit opening the case for repair yourself.

7. Do not let water or other corrosive liquid splash to the machine when using.

8. In the course of the host if there are unusual circumstances, please immediately shut down and unplug the power plug, please contacts the dealer near you or invites the professional maintenance for inspection. Prohibit opening the chassis for maintenance unauthorized

9. If the powers cord damage, we must use a dedicated line or buy the special component from the manufacturer or designated maintenance point to replace

F-312A ULTRASONIC

【F-312A Ultrasonic introduction】

Annoying acne and spots, "high" weight crush many people's pursuit of the dream of the beauty, they can not do anything with a heavy heart. In fact, no need to worry about that, all our problems would be solved by use of ultrasonic skin beauty instrument. Ultrasonic cosmetics has been widely used and popularized since the early of 1980s. Wide therapeutic rage, better effect, high cure rate and easy operation made the ultrasonic beauty instrument the most practical and welcomed among others.

1. Mechanical Function




Ultrasonic wave has stronger power since it has more energy. When it acts on our body, it stimulates the body cells, produces such tiny massage that expands the space in which the cells exist, causes the movement of cytoplasm, the rotation of mitochondria, the vibration of cell nucleus, stimulates and expands the cell membrane. Therefore it can improve local blood and lymph circulation, increase the penetration among the cells, and improve the metabolism and regeneration of body cells.

It is known that the mechanical function of ultrasonic wave can soften body tissues, increase the penetration, promote metabolism, speed up blood circulation, stimulate nerve system and improve, the cells function; therefore the massage applied by ultrasound has more therapeutic function and optimum effect than that by common instrument.

2. Warming and Heating Function

Warming and heating is one of the important therapeutic factors of ultrasound. It is a kind of

internal heat. About 79-80% of which is absorbed and carried away by blood circulating from the action part, while the rest 18-21% of the heat penetrates to nearby tissues. The patients perceive little sense of feeling during the cosmetics.

Different therapeutic method produces different intensity of heat on body tissues. When successive state is used () , higher heat is produced by fixed mode than movable mode; while pulse state is used () , the heat effect will be little; while mixed state is used () , the heat effect will be between two states above.

Generally speaking, the warming and heating function of ultrasound can change the function of blood vessels and change the process of metabolism. Speed up blood circulation reduce nerve excitement, produce sedation and relax the muscles.

3. Chemical Function

The biochemical function of ultrasound mainly comes from its reaction ability of anabolism and of catabolism. Anabolism is a process that centralizes the same and similar molecules. Small doses of ultrasound can promote the synthesis of protein inside the cells, regenerate the wounded tissues; while therapeutic doses, promote the synthesis of fiber cells of human body.

Catabolism is a proceeding that reduces viscosity and quantity of large molecules, so that the concentration of the drug can be reduced and the medicine absorbed.

It is seen that ultrasound has the chemical function of catalysis, accelerate the process of metabolism, changes the PH value of the tissues towards alkali, lightens the skin inflammation suffered from acid poisoning and pain, increases the penetration between cell membrane, decomposes molecules of drugs, helps the drug penetrate into the bacteria body and kill them. In brief, the mechanical, warming and heating and chemical function of ultrasound makes local tissues slightly massaged, the temperature of local tissues raised, expansion of lymph increased, cells stimulated, blood circulation accelerated, metabolism improved, function of the ferment strengthened and PH value changed. All of these result in physiological effect and therapeutic function through the role of nerve and body fluid.

2. Cosmetic Function of Ultrasound

Ultrasound has many beauty function, the effect of medicine has long been proven that the main function is as follows:

1. To make dermal health. Ultrasound can improve the material fibroblast metabolism, so will have normal collagen or elastic fibers, and the result is the flexibility to make leather, the tension increased, skin health
2. To remove the black spot. The role of ultrasonic vibration, can promote melanoma cells left Tam color into the blood vessels and lymphatic vessels, continue to rule out melanoma, the skin will become white ruddy.
3. To remove the facial sores Block. Through Ultrasonic vibration it can to soften the sebum block the pores, so that ruled out smoothly.
4. Slimming. Ultrasound can promote the decomposition of excess fat to lose weight slimming effect.
5. To promote the skin's absorption of nutrients

3、Performance characteristics

1. To meet the comprehensive needs of the body in detail each part, taking into account the different characteristics of the skin, this instrument is specially designed for users with media-size stainless steel probe and small-size stainless steel probe apply to face and eye or a small area of beauty.
2. There are three different forms of output wave, the user can choose to convert, can avoid the use of a single form of long-term role of ultrasound in the human body, so that tissues of the ultrasonic vibration creates an inertia, so that the body can receive a wide range of organizations, the role of care to achieve " the right medicine "in effect, such as inflammation, swelling and

suppuration of the skin capillary tensile pulse should be selected, and the rest can be used continuous-wave or mixed wave.

3. Probes are used of stainless steel materials, stable performance have a good ultrasound transmission properties, so that ultrasonic energy to minimize wear and tear.

【Method of Operation】

A: Connect the output of the probes in to the corresponding function to the label affixed to machines connected and switched on.

Probe connected Zhongping output and a sharp probe into the corresponding function to the label affixed to machines connected and switched on.


B: Turn on the upper left corner of the power switch "POWER" button, then lit LCD display window at the same time.

C: Pre-set the probe, wave, energy and time (The default settings with small-size probe, successive wave five range energy and 15 minutes). You can also change the default settings on the processing.

D: Press START / PAUSE output started work, on the screen shows corresponding ultrasound images with dynamic instructions. Beautician hands probe to the customer site, with the ultrasonic method of massage. It will be able to massage parts of the operation of beauty.

【Panel Instruction】

- ① Power
- ② Probe indicator
- ③ Wave indicator
- ④ Time
- ⑤ Switch button of wave
- ⑥ Energy "+" button
- ⑦ Time "+" button
- ⑧ ON/OFF
- ⑨ Energy
- ⑩ Switch button of ultrasonic probe
- ⑪ Energy "-" button
- ⑫ Time "-" button

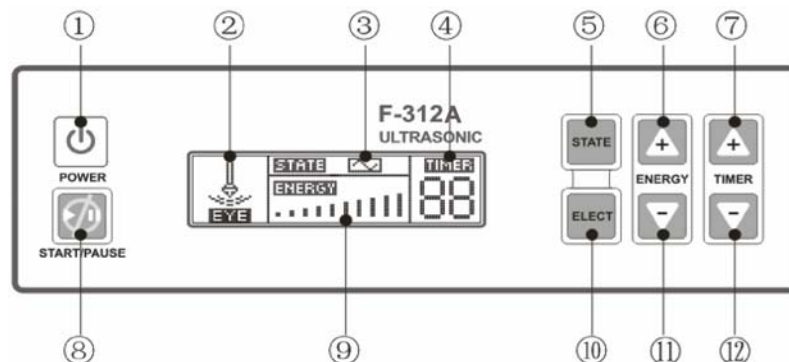

【Notes】

- 1、 To keep clean probe each time the probe should be clean and disinfected with alcohol after us.
- 2、 Not to probe into the eye while using the ultrasound
- 3、 Generally do not care more than 15 minutes, it should decrease the care time on the smaller parts. It will fatigue and skin absorption if use for a long time, the probe should not long remain in the particular skin during nursing process, so as not to damage the skin.
- 4、 If a probe-free output, the probe should first check whether the corresponding output line to connect.
- 5、 If the mainframe can not boot, check the reliability of the power cord connection, followed by the mainframe to check the rear seat with power supply fuse in the fuse positions have been burned, if he has a bad, should replace new 250V/4A safety tube. If they are not above fault, they should contact with vendors to resolve.
- 6、 In the probe to have a facial plane, the energy of "ENERGY" adjustment block more than 7, the

probe easily and avoid the use of heat for too long, the long time operation will not only affect the absorption of skin fatigue, also may be generated by ultrasound mechanical vibration so that the probe temperature.

7、If the power cord damage, we must use a dedicated line or from the factory soft or its maintenance department to buy a special component to recommend the use of original replacement parts.

【Warnings】

- 1、Do not leave the detector unattended while it is outputting, otherwise, may quickly shorten its lifetime for over heated.
- 2、Do not attempt to separate the stainless steel detector and the handle, any attempt to fasten may cause detector damages.
- 3、Forbid to put the probes in water to clean, which may cause damage to the probe.
- 4、Forbid to use the damage of the probe with surface scratches, it must replace the probes.

F-313A GALVANIC

【F-313A Galvanic Instruction】


By mean of the movements of DC positive and negative ions, the galvanic current treatment is helpful for leading in nutrition and leading out impurity from skin. The nutrition leaded in is to penetrate the water-soluble nutrition into the deep skin, directly supplying nutrition for skin. Impurity leaded out is to import the alkaline solution to soften the aged horn, decompose the grease balance the skin secretion, leading out the impurity and metal ions at the same time; The electronic ions of positive electrode may lead the acidic solution (shrinking lotion) into the skin, and harden the skin tissue, comfort nerve and diminish blood supply as well. It is mainly used after beauty treatment to alleviate skin being red and swollen and to avoid inflammation. The electronic ions of negative electrode may lead alkaline solution into skin to soften skin tissue, irritate nerve and increase blood supply. Mainly used in aged and greasy skin.

【Galvanic Operation Instruction】

1. Connecting the accessories corresponding to the instrument before turning on the power. Client grasps the metal pole, and beautician holds the handle of the roller.
2. Changing the electrode of the roller by adjusting leading in (+R) and leading out (-G). The treated part must be clean and keep it clean and dry before doing the nutrition leading in. Turn the electrode button to (+R), now the roller is positive while the metal pole is negative. Beautician spreads some liquid nourishing lotion evenly on the treated part, then cover the roller with a piece of cotton or sponge that has dipped into the acidic lotion (like shrink lotion) or directly cover the roller with a piece of cotton with nourishing lotion before doing this treatment. While doing leading out treatment, should change the electrode button to (-G), now the roller is negative while the metal pole is positive. Cover the roller with a piece of cotton or sponge that has dipped into the alkaline lotion (like detergent lotion) before doing this treatment. It is to balance and decompose the grease of skin.
3. Power on "IONOS", select the operation form: leading in "+R" or leading out "-G", turn "I/Energy" to the minimum. Touch the treated part with the roller, slowly adjust "I/Energy" to appropriate intensity which client feels mildly stung. Smoothly slide the roller, and the cotton must keep moisture all the way for better effect.

【Panel Instruction】

- ①POWER
- ②Indicator for galvanic polarity
- ③Polarity
- ④TIMER
- ⑤ENERGY“+”button
- ⑥TIMER“+”button
- ⑦ON/OFF
- ⑧ENERGY
- ⑨Switch button of galvanic probe
- ⑩ENERGY“-”button
- ⑪TIMER“-”button

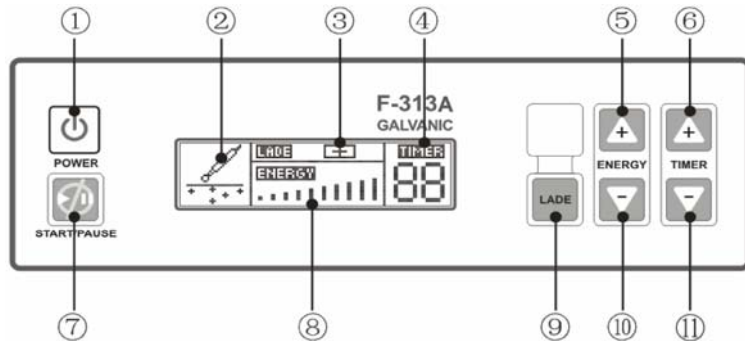

【Cautions】

1. After treatment, the roller and metal pole must be clean and keep in dry to avoid damage.
2. When the roller is negative, the stung feeling will be felt more intensive than it is positive, otherwise, it will be felt weaker.
3. Before doing the treatment, clients will be required to take off jewels, it is not applicable for someone who there are any metal objects inside.
4. Pregnant women and people who have sensitive, injured, inflamed skin or blood capillary expansions are not allowed for the treatment.
5. The lotion with leading in or leading out must be used in accordance with the roller (negative or positive). While the roller is positive, the lotion should be acidic (+), while the roller is negative, the lotion should be alkaline (-). Please use the PH paper for verification if the lotion is not indicated.

【Warning】

1. With the following diseases, not suitable for the use of this instrument, such as heart disease, infectious disease patients, allergic patients and the patients who have metal in body during therapy period.
2. Don't let the leading-out probe and leading-in probe touch together for a long time when they are working.
3. When the lotion of the skin surface cannot moisten skin, it should be added in time.
4. Prohibit to use the sliding wheel which can not rotate, this accessory must be changed
5. Please pay attention to the connecting wire between the machine and accessory when working, don't over-pull it.
6. Do not let water or other corrosive liquid splash to the machine when using.
7. In order to avoid scratching skin, prohibit to use the bead probe and roll probe which the surface of stainless steel is defected、rusted or polished. The export-in and export-out probe should be protected carefully, don't let other hard objects scratch the probes.
8. In the course of the host if there are unusual circumstances, please immediately shut down and unplug the power plug, please contacts the dealer near you or invites the professional maintenance for inspection. Prohibit opening the chassis for maintenance unauthorized
9. If the powers cord damage, we must use a dedicated line or buy the special component from the manufacturer or designated maintenance point to replace.

F-314A High Frequency Electrotherapy

【F-314A Theory of High Frequency Electrotherapy】

1、High Frequency Electrotherapy instrument generates high speed vibrating current, which may convert to heat energy and kill bacteria. As the vibration is too fast, it won't cause muscle shrinkage. And it gives physical stimulation.

Function:

- a. Accelerate the blood circulation.
- b. Stimulate the glandular activity.
- c. Help the skin for excretion and absorption.
- d. Promote metabolism.
- e. Kill bacteria
- f. Generate heat energy inside skin tissue.
- g. Help beauty cream penetrate into deep skin

【Instructions】

First choose a proper glass electrode pipe and insert to the output mouthpiece of electrotherapeutic pole handle, then connect it to the instrument.

Glass electrode pipes assignment position:

- A. Mushroom-shape electrode pipe---- used for large area of skin: face, forehead, neck, etc.
- B. Ladle-shape electrode pipe---- used for medium area of skin: chin, etc.
- C. Dot-like electrode pipe---- used for small area of skin: nose, etc.
- D. Straight electrode pipe---- used for indirect treatment.

1、Direct usages:


- A: Beautician grasps the handle of hi-frequency electrotherapeutic pole (try not to touch glass electrode pipe with hands or metal objects), and turn “H/Energy” button to minimum.
- B: First power on “Hi-Frequency”, and turn “H/Energy” button to minimum, then touch the body part that is to be treated with glass electrode pipe, slowly adjust the “Hi-Frequency” button clockwise until mild hi -frequent current will be felt. If the face is covered with cream or mask, move the glass electrode pipe upward from client's neck, slowly slide to chin, face, cheek, nose and forehead. In the course of treatment, you should slowly adjust the “H/Energy” to the appropriate intensity. As the skin tissue generates heat energy, it may comfort and calm the skin by directly using the Hi-frequency Electrotherapy; meanwhile it is efficiency for killing bacteria. It is applicable to oily and with acne skin.
- C: After treatment, slowly twist “H/Energy” button to minimum, and power off “Hi-Frequency”, and take away the glass electrode pipe from the client's face.

2、Indirect usages:

First let the client hold the straight electrode pipe, and power on the “Hi-Frequency”, gradually turn the “H/Energy” to the desired intensity, then the beautician massages client's treated part. In the whole process, the beautician does not touch the electrode. The beautician must power off before taking away the glass electrode pole from the client. Indirect usages will accelerate and comfort the skin at the same time.

【 Panel Introduction 】

- ①POWER
- ②High-frequency Indicator
- ③ENERGY
- ④TIMER
- ⑤ENERGY“+”button
- ⑥TIMER“+”button
- ⑦ON/OFF
- ⑧ENERGY“-”button
- ⑨TIMER“-”button

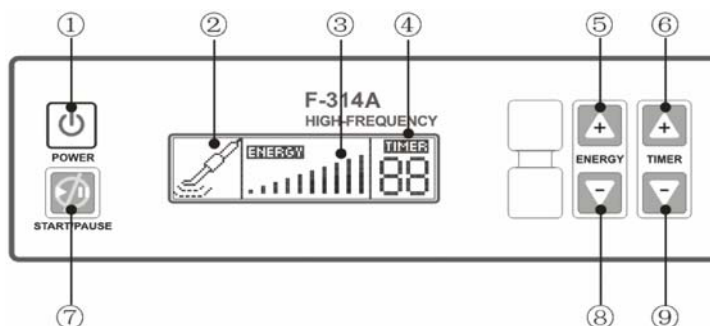

【Cautions:】

1. The glass electrode pole will sparkle when it is still in less than 1 inch distance from the skin while the instrument turning on, please reduce the energy of Hi-Frequency. If the client still feels the sparkle existing, may cover the face with a small piece of dry cotton towel but not the nylon or synthetic fiber one.
2. For preventing electric shock, beautician must choose the minimum intensity before touching the client's face, and slowly increase the Hi-Frequency energy.
3. Before taking away the glass electrode pipe from the skin, must twist the "H/energy" to minimum, then power off and take it away.
4. When loading/unloading the glass electrode pipe, must turn the "H/Energy" to minimum, and power off "H/Energy".
5. Glass electrode pipes must be kept clean and dry, and sterilized before or after use.

【Warning】

1. When using the treatment of High-frequency, customers can not come into contact with any metal objects, if worn, must be removed.
2. Pregnant and the person who has pacemaker, can not use high-frequency electrotherapy.
3. Do not use the cream and mask which contain alcohol when using high-frequency electrotherapy. If you must use lotion containing alcohol, please after applying high-frequency electrotherapy.
4. Do not face the various electrode tube right in the eyeball.
5. Do not touch the output interface or handle output interface of the high-frequency electrotherapy machine when it is in the state of charge.
6. This equipment should be operated by professional beautician, there will be a lot of interference by high-voltage current output when working, be careful not to close other electrical appliances, including the host machine. If we do not care to interfere with the host, the host maybe does not work or may lead to system halted. Let the host power off and restart, the host will return to normal state. If not improved, maybe the interference is too much that conduce the damage of host parts.

F-315A Rotary Brush

【F-315A Function】

By means of the rotating force of the brush, it can massage the skin. It utilizes circumrotation of brush to massage, deep clean the skin and wipe off cornea, which obtain improving skin quality stimulating blood circle and removing fatigue. Moreover, its rhythm of massage can gain elasticity of skin and smooth skin.

【Using Method】

1. Plug the connector of brush handgrip to the port back of the host, hold the brush handgrip by

the petiole part.

2. Choose appropriate brush head to insert the handle (5 brush heads are available, can be used at different parts of body).


Press the “POWER” key on the host, the LCD works; The host comes into the default mode; The direction of rotary brush is “↺”; The time is “3 minutes”; And the energy is “3”.

Press “START/PAUSE” key to start the rotary brush, then the cosmetology can be started. A dynamic brush logo will be displayed on the LCD when the host begins to output. The dynamic brush logo becomes stable when the host stops output.

3. Depends on the demands of cosmetology, the rotary direction can be changed by pressing the “⑨” “DIRECTION” key; the working time of host can be adjusted by pressing “TIMER” “+” “-” keys; The rotary speed of the brush can be adjusted by pressing the “ENERGY” “+” “-” keys.

【Illustration of panel】

- ① Power
- ② Brush working indicator
- ③ Direction indicator
- ④ Time indicator
- ⑤ Energy increase key
- ⑥ Time increase key
- ⑦ Start/Pause
- ⑧ Energy indicator
- ⑨ Direction switch key
- ⑩ Energy decrease key
- ⑪ Time decrease key

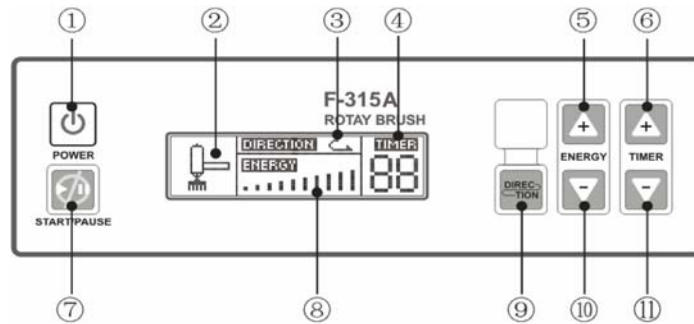

【Cautions】

1. Before working please check if the brush head has been plugged properly on the handle.
2. Before the brush head touch the face of the customer, the rotary speed should decrease; don't make the customer feel uncomfortable.
3. After working, all the brush heads should be cleaned, sterilized and keep in dry place.

【Warning】

1. The function can not be used for long time, otherwise, the handle will be overheated.
2. The brush head can not be exchanged under the working condition.
3. The brush head can not be stopped by hand or other objects under the working condition; and grasping the brush head when starting the function is also forbidden.

F-316A Vacuum & Spray Instrument

【F-316A Vacuum & Spray Functions】

If black-head, white-head and acne grow in the face, they not only influence the appearance, but also block the pores. When they become worse, they will block the blood circulation, and influence the skin's metabolism. After the influence cumulates for a long period, the pores will become bigger and bigger, and the skin will become rougher and rougher. This instrument can suck the black-head, white-head, acnes, blains and clean the besmirch and oil in the pores effectively. It can also spray the fog of astringent, diminish inflammation water and shrink water. It can shrink the pores effectively and has the functions of diminish inflammation, easing pain and supplement nutrition.

【Usage Method】

1. Black-head vacuum: Connect the single air pipe to the corresponding vacuum pipe marks nozzle, the other head to the small glass vacuum pipe. Three small glass vacuum pipes for option to perform black-head vacuuming.

Turn on the power of “VAC-SPRAY”, and adjust the proper energy and time. Then press the “START/PAUSE” to start work.

It is advisable to stuff some absorbent cotton into the glass vacuum pipe for stopping dirt going into the pipe so as to prevent damaging the instrument. There's a small hole in the front end of the vacuum pipe, firstly stop the hole by finger as the illustration, when the vacuum pipe touches the face the finger can move away or just move partly to control the vacuum force.

The glass vacuum pipe must not stay on the face too long so that it will scald the face. It is normal if there is no vacuum at the beginning, it will resume to normal condition after a while.


2. Spray: Connect one head of the single air pipe to the corresponding spray bottle marks nozzle, the other head to the small spray bottle.

Turn on the power of “VAC-SPRAY”, press the “START/PAUSE” key to start work, and adjust the energy and time properly.

Two spray bottles that can contain different liquid, it will spray out by pressing the nozzle on top of the spray bottle with finger, spray downward from forehead during working, prevent liquid going into the nose, ask client to close eyes and shut mouth. Keep a adequate distance from the face.

【Illustration of panel】

- ① Power
- ② Vacuum & Spray indicator
- ③ Energy indicator
- ④ Time indicator
- ⑤ Spray working key
- ⑥ Energy increase key
- ⑦ Time increase key
- ⑧ Start/Pause
- ⑨ Vacuum working key
- ⑩ Energy decrease key
- ⑪ Time decrease key

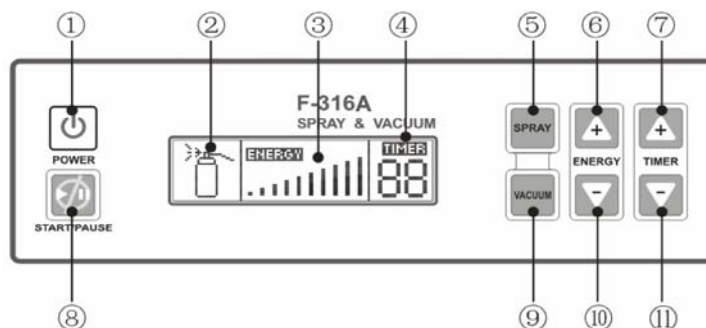


【Notes】

1. Before using the function of vacuum, unscrew the glass vacuum pipe to check if there's absorbent cotton. The absorbent cotton can stop dirt going into the pipe so as to prevent damaging the instrument.
2. The absorbent cotton in the glass vacuum pipe must be checked or replaced before usage each time. The glass vacuum pipe will be blocked if there is too much dirt accumulates on the absorbent cotton.
3. After working, all the air pipes and accessories should be cleaned, sterilized and keep in dry place.

【Warning】

1. It is forbidden to insert the Spraying tube into the Vacuum sticker collector, in order to prevent the dressing water is sucked into the pump to damage the machine.
2. When using the spray function, it is forbidden to plug up the output connector of “Spray” by anything or fold the silica gel pipe, this will damage the host.
3. The broken or cracked glass vacuum pipe is strictly prohibited to use.
4. Don't pull the silica gel pipe roughly, or it will damage the pipe.

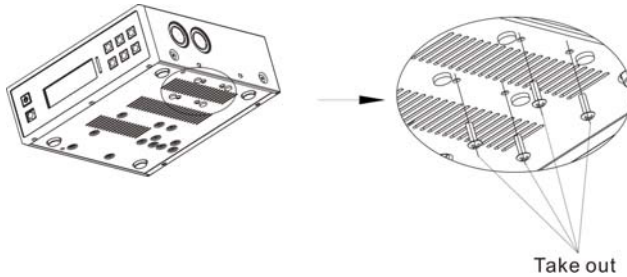

F-317A Magic Glove


【Introduction of F-317A Magic Glove】

Magic glove applies special low voltage pulse currents on human body, to activate skin cells, promote metabolism, restore skin elasticity and vigour.

BIO-skin smoother stimulate the skin metabolism with bionic signal, which is very helpful in improving the contractibility of fibrous tissue, activating cells, enhancing the anabolism of collagen and tropoelastin, dilating blood vessel, discharging the skin metabolin. BIO-skin smoother makes the loose skin tighter, smoother, and more elastic.

【Galvanic Treatments】


- 1、Insulative disposable gloves must be put on before the beautician puts on Magic Gloves. (The requirements of disposable gloves refer to the first point of cautions.)
- 2、Connect the main set and accessory lines well. Connection refers to the figure below.

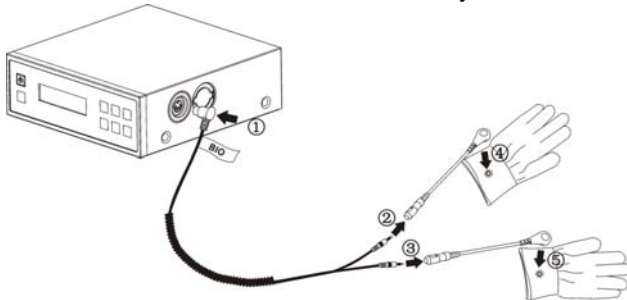

(The five points shown above should be well connected)

【Panel Introduction】

1. Power switch
2. Frequency indicator
3. Time indicator
4. Frequency button
5. Energy increasing
6. Time increasing
7. Start/Pause button
8. Wave indicator
9. Energy indicator
10. Wave switch
11. Energy descending
12. Time descending

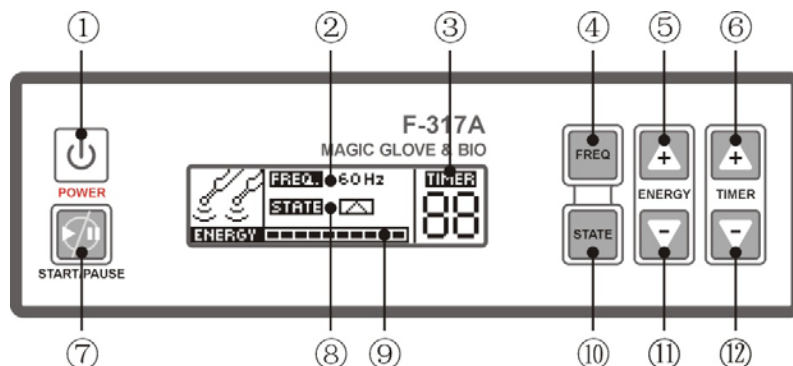

【Notice】

- 1、The standard length of this set of accessories magical gloves is 20 cm, the length of disposable gloves should be no less than 23 cm. Magic Glove might contact the beautician hand if the disposable gloves are not long enough, which may cause defective beauty result.
- 2、After the treatment, magic gloves should be cleaned, sterilized, dried in the air then kept in dry place.
- 3、No treatment on epileptic and chemicals taker.

- 4、No treatment on those with sensitive skin, injured skin, dermatitis, and who are pregnant.
- 5、Before treatment, please take off all jewelry and metal pendant from customer; No treatment on those implanted metal advice.
- 6、Please renew degraded accessories after long term usage.

【Warnings】

- 1、No treatment on patients of cardiopathy, communicable diseases, nerve allergic or those implanted metal advice.
- 2、Don't put the working magic gloves together continually. An over-one-hour direct conduction of the working gloves may shorten the service life of main set.
- 3、Be careful with the connected wires between accessories and main set during treatment. And try not to pull.
- 4、Main set should be avoided from water or caustic liquid.
- 5、Please unplug the main set and get contact with local distributor or professional technician in case the machine encounters any abnormal condition. Professional repair only.
- 6、To replace power line, please purchase from manufacturer or nominated customer service center.

F-318A Cold & Hot Hammer


Brief introduction

1. With the principle of cold & heat conversion, the product uses special refrigeration film, could convert energy to the temperature you need in short time.
2. Can activate cell, improve and purify skin quality; smooth wrinkles and crow's feet more naturally and thoroughly.
3. The alternate use of cold and heat, could expand and shrink pore fastly, give them movement of expansion and shrinkage, thus give the skin vitality and vigor for the elastic.

Operating Instruction

1. Connect host engine and accessories lines well.
2. After press the power switch, first set cold therapy or hot compress function; temperature and time again; and then press the "START/PAUSE" to start the host; last, open the switch on the handle of the hammer.
3. For first use, it takes about 5 minutes to work normally after complete the setting.
4. When one treatment is finished, 3 minutes is needed to switch to another.

Panel Instruction

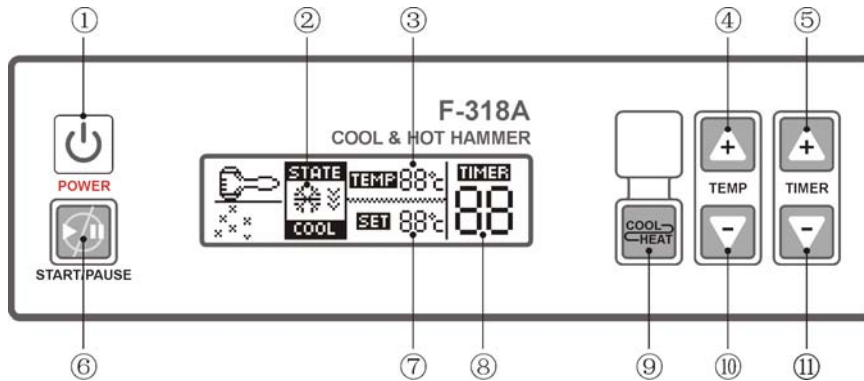

- ① Power Switch
- ② Cold/Hot Indicator
- ③ Real-time Temperature Indicator
- ④ Temperature Increasing Button
- ⑤ Time Increasing Button
- ⑥ Start/Pause Button
- ⑦ Setting Temperature Indicator
- ⑧ Time indicator
- ⑨ Cold/Hot Switch Button
- ⑩ Temperature Decreasing Button
- ⑪ Time Decreasing Button

Cautions:

1. Indoor use only, because the working condition of this instrument would change as the environment temperature changes. The instrument probably can't work well if the temperature is too high or too low, the best is to operate in the room with the temperature about 25°C
2. The scattered gas tank shouldn't be blocked. if so, some condensed water may produced inside the hammer, thus may cause short-circuit fault.
3. When the host engine shows in the state of working after starting the "START/PAUSE", but the cold & hot hammer still don't work, pleas check whether the switch of the hammer has been adjusted to "-" or not.
4. Besides the time of primary seeting starting time and switching cold/hot function, you also need to add the time of temperature going up/down (start time: 5 minutes; switch cold/heat: 3 minutes).

Warnings:

1. The two ends of the cold & hot hammer are made of aluminum alloy, they have soft hardness, so couldn't be used as hitting tools.
2. Scratches arise on both ends, new accessories need to be replaced for next use.
3. Don't pull the spring line too much during the process of using, pay attention to the operating range.
4. The host engine and cold & hot hammer shouldn't be tear apart privately; If there are any unnormal, contact the distributor for maintanance please; open the case to repair personally is

prohibited.

5. When arise the following cases: the powerline or plug of the host engine/ cold & hot hammer in the attachment is damaged, tured over, damaged or get wet by the water, don't continue to use the machine again. Please send the machine back to the service centre for check and maintain.

6. Using in damp environment is banned, such as bathroom, etc.


F-319A User Manual

【Function】

1. Eliminate the aging stratum corneum, tender skin, clear away dirt from pores, fade acne scars;
2. Suck accumulated oil from the skin and the pore, improve dull skin, eliminate facial swelling, eye swelling;
3. Increase skin elasticity, eliminate slender wrinkles, improve skin texture, keep the skin youth;
4. Improve the skin from dull, rough, uneven complexion, pigment, spots;
5. Dilute the tattoo pigment, fat pattern, stretch marks;
6. Eliminate the aging stratum corneum, tender skin.

【Illustration of panel】

- ①Power
- ②Energy indicator
- ③Time indicator
- ④Energy increase key
- ⑤Time increase key
- ⑥Start/Pause
- ⑦Energy decrease key
- ⑧Time decrease key

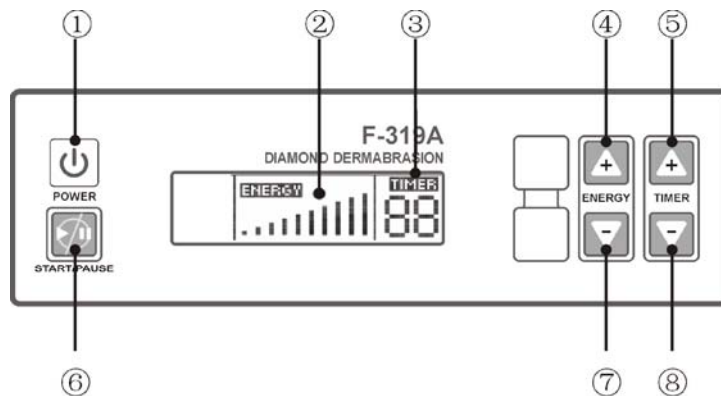


【Fundamental】

This Instrument is made use of a mosaic of different thickness diamond particles, with controlled grating, and matched up the vacuum suction to control strength. It makes partial miniature rejuvenation for the lymphatic drainage, skin aging, pigmentation or large pores, etc and brings the most gentle and rapid regeneration for the skin. Not only can accelerate the renewal of the aging stratum corneum, but also increase the full absorption of skin care products.

【Features】

1. The newest high-tech diamond dermabrasion rejuvenation technology, security, painless, no side-effects;
2. 9, Diamond Head: 3 pins for different facial skin, 2 specialty pins for eye and slender wrinkles, 4, specialty pins for the body with a wide range professional treatment
3. Humanization design, developed operating handle elaborately, accord with hand structure and facilitate the beauticians;
4. With simplicity of operation, it makes the skin silky, natural, safe effectively;

5. No any powder in the process, to avoid the powder residue on the skin or suck by the guests or beautician;
6. Needn't rest after treatment, you can go out or make-up, it wouldn't affect the normal social life.

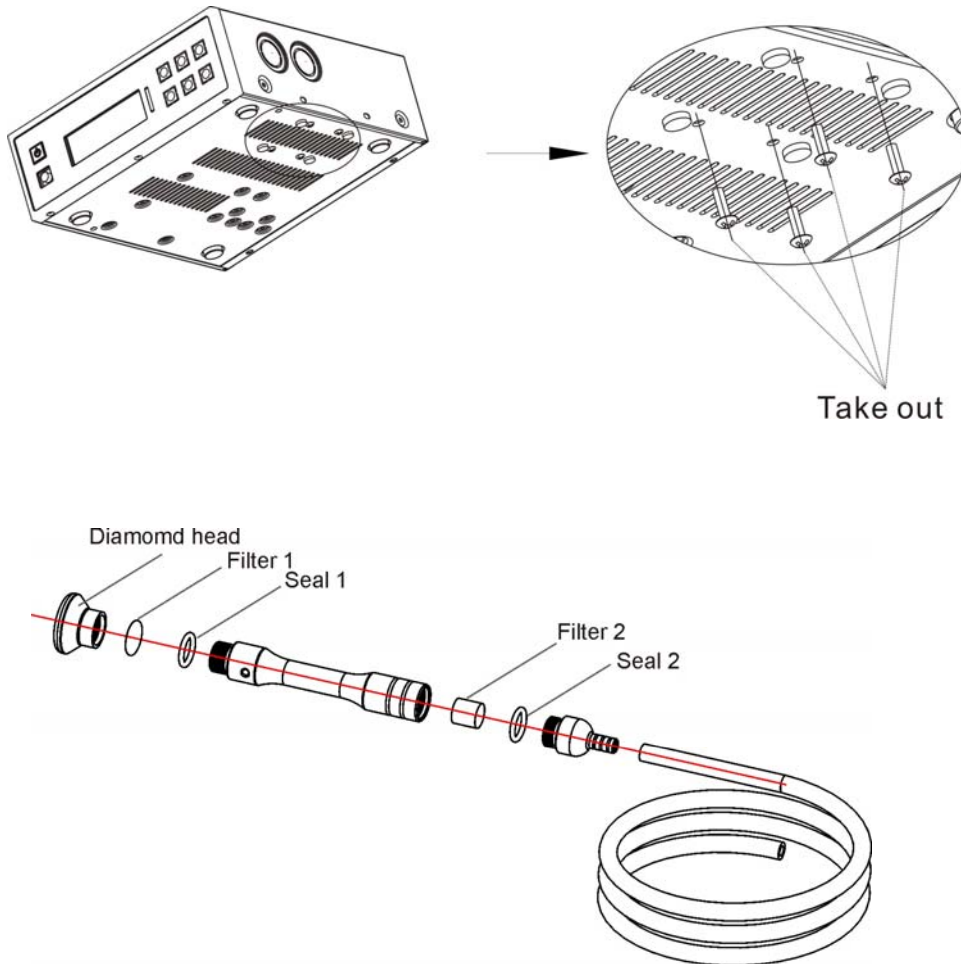

【T-01 Hot Towel Cabinet Instruction 】

- 1、 Simple construction, fashionable design; it is durable by using the metal door and the new metal door handle.
- 2、 Aluminous inner case with adopting one molding technique, firm and beautiful.
- 3、 Outer case use the high quality material with the high temperature protective powder spray in appearance, causticity prevent, and good looking.
- 4、 Host has double protective functions; It has independent switch for ultraviolet light. When the door is opened, the ultraviolet light will automatically shut down. The ultraviolet light can go on

- working after the door is closed.
- 5、The special glass window construction can prevent the ultraviolet light leaking and inspect the conditions in the inner cabinet easily.


【Working Theory】

Heater system:

It can transfer the electricity to heat energy by using the special heater deliver setting. The heater energy can be transfer by infrared ray. This system use complete self-motion temperature setting to raise the heater energy after pressing the heat preservation switch. When the temperature reaches the setting temperature, the system will move to the heat preservation state. When the inner temperature is lower than the setting temperature, the heater system will operate to raise the temperature automatism.

Sterilizing system:

Press “OZONE” switch to operate the sterilize functions, the system will operate the ultraviolet light and ozone setting to sterilize the towel. Forbid to open the door, or the system will shut down the sterilize functions immediately to avoid the ultraviolet light leaking.

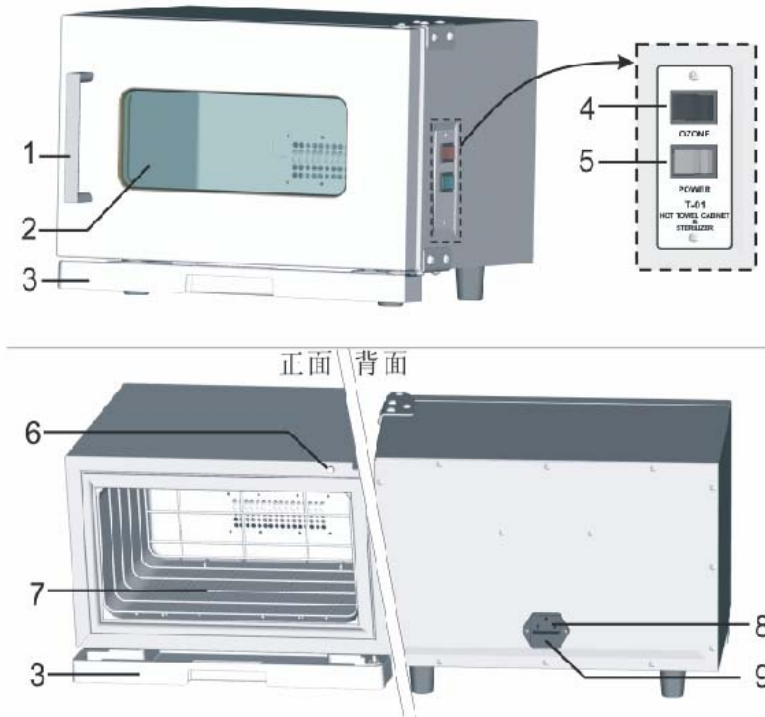

1. Door handle: Open and shut the door.
2. Glass window: Inspect the inner article.
3. Water plate: Collect all the dripping water in the door.
4. Ozone switch: Control the ultraviolet light and ozone operation.
5. Power: Control the power connection.
6. Gating switch: When the door is opened during working, the ultraviolet light and ozone operation will be shut down automatically by this button.
7. Towel bracket: Place the towel.
8. Power line plug: Main power input connector.
9. Fuse seat: Install the host fuse.



【Operation】

1. Connect the instrument with the host, as the illustration on the right.

2. Open the door and put the towel regularly in the towel bracket for sterilizing. Shut the door and inspect the seals is connected well or not. Press the

“POWER” switch to operate the instrument for sterilizing. When the temperature reaches 60 degree, the instrument will move to the constant temperature state; you can press the “OZONE” to operate the ultraviolet light and ozone function. The instrument begins to sterilize when then ultraviolet light is on in the inner cabinet.

3. Clean the water in the water plate: Draw out the water plate by using its hand gripe. Push it back in the contrary direction after cleaning the water in water plate. (See the illustration)

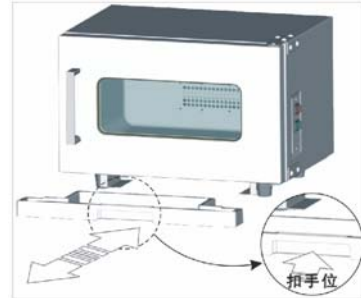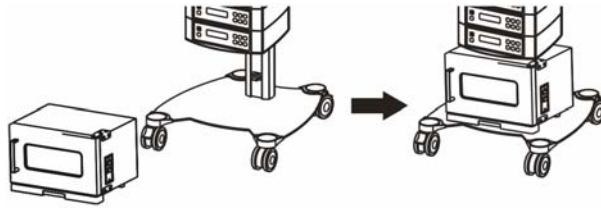

【Notes】

1. Forbid to use the hot towel cabinet nearby the high temperature object.

2. Forbid to use the hot towel cabinet in the acid air or in the humid environment.

3. Forbid to put the acid or alkaline articles into the hot towel cabinet.

4. Forbid to move the towel cabinet on power or on working.

5. It is normal state when the outer case has obvious temperature raising after long time using.

6. Please clean the water leaving in the water plate regularly after you doing the heat preservation for wet towel. (Clean it every day will be better)

7. The sterilizing time is better to be limited in 1-2 hours that you are using the ultraviolet radiation and ozone sterilization.

8. Please shut down the power and take out the wet towel or other articles that you don't sterilize any more. Please keep the inner case dry by wiping the water with dishcloth. Please draw out the power plug if you will not use the cabinet for a long time.

9. Please keep your hand dry when pressing “POWER” or “OZONE” switch.

10. Please contact the dealer for exchanging the glass if the glass of hot towel cabinet is broken. The glass is made by special technique that can avoid the ultraviolet light leaking. It cannot be replaced by the usual glass.

11. Please contact the dealer or professional maintenance office to repair the hot towel cabinet if it has anything wrong.

1005 Cool Magnifying Lamp

【1005 Cool Magnifying Lamp manual】

This machine mainly use the magnifying function of the magnifier and cold lamp, use them to check the various small transformations of the skin surface, can timely find out the different conditions, through careful treatment and maintaining so as to keep the skin slick, delicate and stretchable.

With the unique design and magnifying function, it is also can be used in the beauty items like tattoo, grafting the eyelashes and so on.

The magnifier can be used together with the bracket, or it can be used together with other instruments.

Novel design, pretty appearance. Its bracket has the protector to prevent nipping hands during operation. The magnifier adopt the lens with the usual 5 times magnifying rate and partially 8 times magnifying rate together. The soft lamplight will not make the user feel dizzy.


【Usage Methods】

- (1) Insert the electrical plug.
- (2) Turn on the power switch, turn the magnifying lamp towards the skin surface which is to be checked or operated on.
- (3) Adjusting the distance between the magnifying lamp and skin, and achieve the most clear condition.
- (4) Check the skin or start the beauty operation.


【Usage Cautions】

- (1) Forbid to use the instrument in the over humid environment, like bathroom, etc. Clean the skin surface before the beauty operation.。
- (2) Turn off the power after the operation is finished.
- (3) Please draw out the plug if you won't use it for a long time. Wipe the lamp cleanly and cover it then place it inside the corner.
- (4) Please use the soft lens paper to clean the dust on the lens. Don't use other cloth or paper to clean it, otherwise the lens will be damaged.
- (5) Don't disconnect the instrument by yourself, please ask the professionals to test or repair it if any mistake occurs.

【Installation of magnifier and dolly】

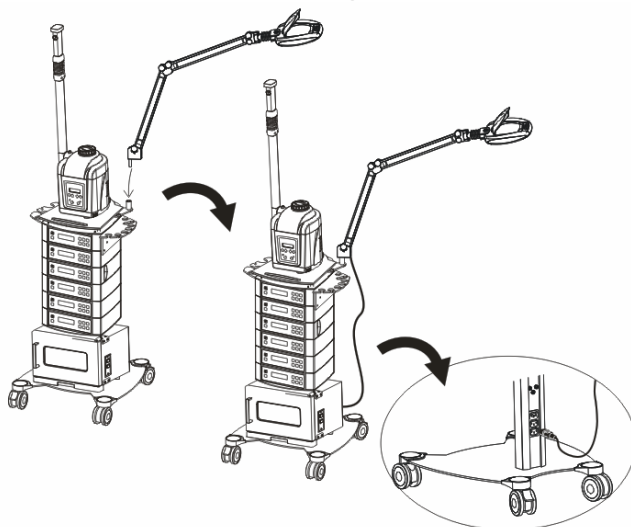


【Installation of the functional instruments' accessories】

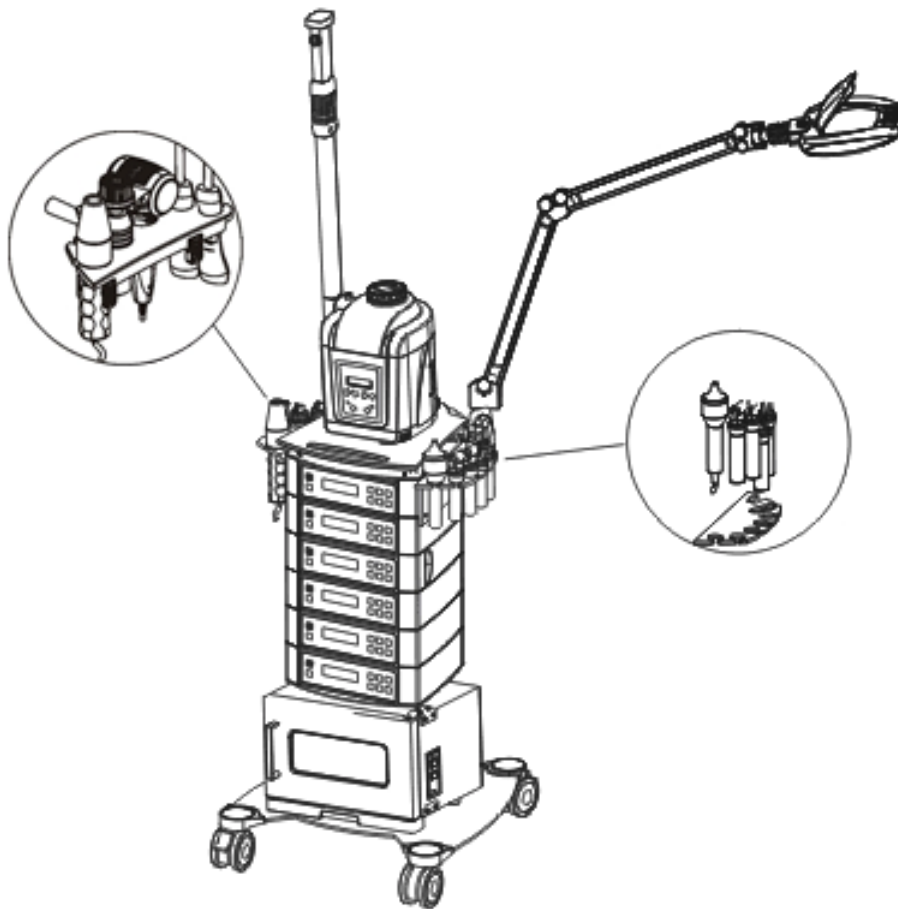


【Installation of the cables】

The accessories' cables must be connected with the right instruments and right plugs. The host and accessories can be permanently damaged if the cables are wrongly connected.

The label on each cable must be corresponded with the plugs. See the following illustration.

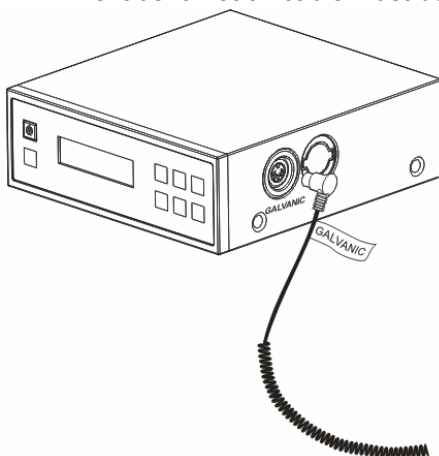


The accessories' cables must be connected to the host through the holes in the bottom board. Otherwise the cables will stop both doors when you close them.

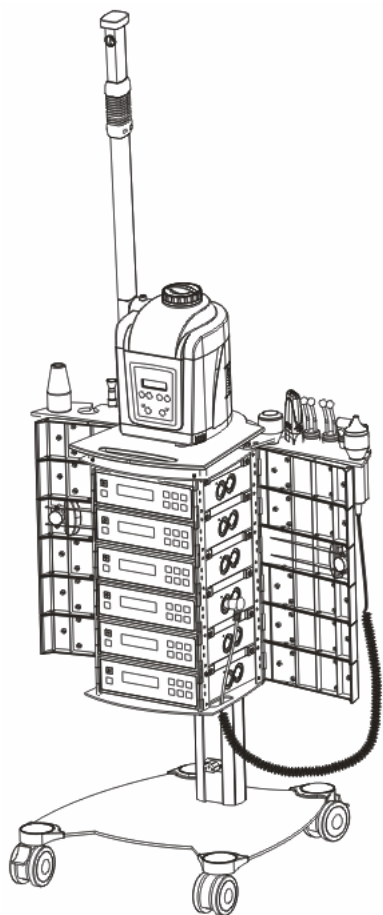

【Power Connection, Input and Output】

The first triple socket is for power input. The other sockets are for power output of Magnifying Lamp and Spray Instrument.

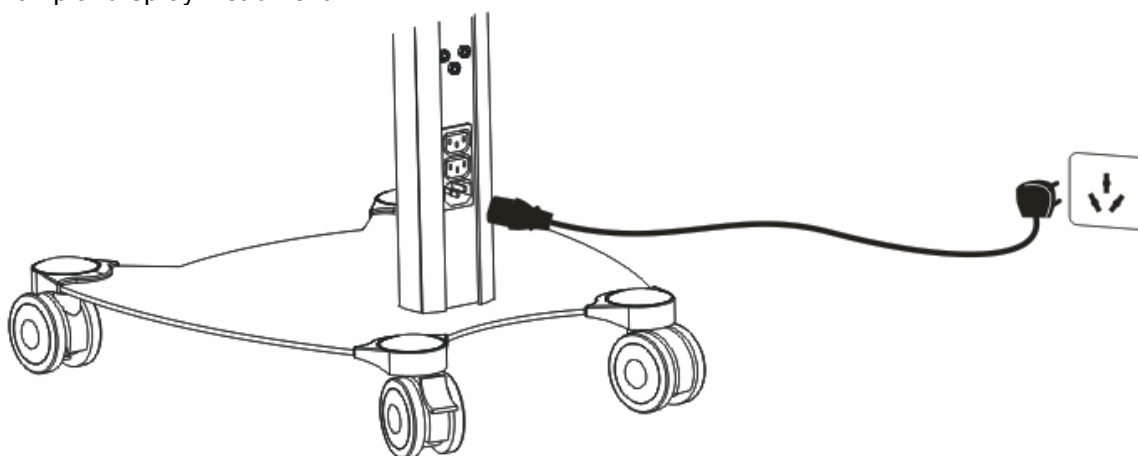

Item & Technical Parameter

Name	Hot Towel Cabinet	
Item no.	T-01	
Rated working Voltage	AC 100-130V	AC 220-240V
Rated working Frequency	50Hz	60Hz
Rated working power	164W	
Capability of container	7 L	
Measurement (length×width×height)	330×276×226 mm	
N.W.	4.5 Kg	
G.W.	5.5 Kg	

Name	Facial Steamer	
Item no.	F-300DT	
Rated working Voltage	AC 100-130V	AC 220-240V
Rated working Frequency	50Hz	60Hz
Rated working power	800W	

Name	Magnifying Lamp	
Item no.	1005	
Rated working Voltage	AC 100-130V	AC 220-240V
Rated working Frequency	50Hz	60Hz
Rated working power	22W	

Name	Multiple Beauty Instrument	
Item no.	F-311A	
Rated working Voltage	AC 100-130V	AC 220-240V
Rated working Frequency	50Hz	60Hz
Rated working power	20W	
N.W.	3.5kg	

Name	Multiple Beauty Instrument	
Item no.	F-312A	
Rated working Voltage	AC 100-130V	AC 220-240V
Rated working Frequency	50Hz	60Hz
Rated working power	20W	
N.W.	3.8kg	

Name	Multiple Beauty Instrument	
Item no.	F-313A	
Rated working Voltage	AC 100-130V	AC 220-240V
Rated working Frequency	50Hz	60Hz
Rated working power	20W	
N.W.	4kg	

Name	Multiple Beauty Instrument	
Item no.	F-314A	
Rated working Voltage	AC 100-130V	AC 220-240V
Rated working Frequency	50Hz	60Hz
Rated working power	20W	
N.W.	3.5kg	

Name	Multiple Beauty Instrument	
------	----------------------------	--

Item no.	F-315A	
Rated working Voltage	AC 100-130V	AC 220-240V
Rated working Frequency	50Hz	60Hz
Rated working power	20W	
N.W.	3.6kg	

Name	Multiple Beauty Instrument	
Item no.	F-316A	
Rated working Voltage	AC 100-130V	AC 220-240V
Rated working Frequency	50Hz	60Hz
Rated working power	120W	
N.W.	7kg	