

SILVERFOX

F-312B

Beauty Instrument

User Manual

Welcome to use our products. A0
Please read the user manual carefully before using the products.
Our company reserves the right to improve design and modify
the products without prior notice.

About this manual 1

Safety Guidance 1

F-312B Ultrasonic’s Introduction & Parameters 2/3



Troubleshooting 4

About this manual


This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, SILVERFOX will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of our company SILVERFOX.

Safety Guidance


**To ensure your correct and safe use of this product, please read all the instructions before use!
Please follow all safety precautions!**

	Warning!	<p>Indicates a potential dangerous situation that, if not avoided, could result in serious loss of life.</p> <ul style="list-style-type: none">• This product is only for beauty practitioners with professional qualifications.• Do not use or store the product in a damp place such as bathroom to avoid malfunction and danger.• Do not use this product near flammable items (e.g., alcohol, thinner, gasoline, nail polish remover, sprays, etc.).• Do not use this product under the following conditions: when the power cord or plug is damaged or gets hot; when the plug in the power socket comes loose.• Do not use wet hands to plug and unplug power cord connectors and use this product.• Please make sure that the working voltage of this product conforms to its rated voltage.• Before use, make sure that the power plug and the power cord connector are fully plugged in, so as not to cause danger.• When not using this product, please turn off the power and pull out the plug.• Please regularly clean the dust accumulated on the power plug and power cord connector to avoid failure of insulation caused by moisture, dust and other factors.• Be sure to use the special power cord assembly for this product. Do not use the power cord assembly on any other appliance.• Do not wrap the power cord around the product, otherwise it may distort and malfunction the power cord.• Keep out of reach of children. Do not allow children to use this product.• Please do not modify, disassemble or repair this product by yourself. If you need to repair it, please contact the manufacturer, agent or professional maintenance personnel to check and repair it, so as not to cause danger.• Please contact distributor, service center or related personal with appropriate skill and experience for repairing if the plug is damaged, in case of any danger.• Please be sure to use this product according to the purpose specified in this manual. Do not use accessories not recommended by the manufacturer.
	Attention!	<p>Represents a potentially dangerous situation which, if not avoided, can result in general injury to person.</p> <ul style="list-style-type: none">• Operators should pay special attention to the safety of personnel and equipment when using.• If the product is abnormal, please stop using it immediately and contact the manufacturer or dealer for relevant treatment.• This product and related accessories should be handled lightly.• Please properly handle the packaging bag of this product to avoid accidents such as asphyxiation of infants, and keep it out of reach of children.

◆ F-312B Ultrasonic (Only used for beauty)

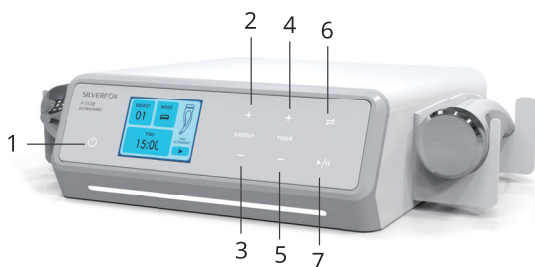

1. Power Switch
2. Power Energy Increase +
3. Power Energy Decrease -
4. Time Increase +
5. Time Decrease -
6. Mode Switch
7. Start/ Pause

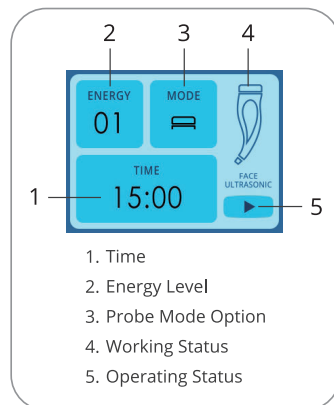

Operation Method

1. Insert the Ultrasonic probe into the corresponding socket of the Equipment. When the equipment is connected to the power cable properly, the "Power switch" is displayed in red. Press the "Power Switch" to enter into the working mode and it would show in white on the screen. The equipment defaults to the state of eye probe output; thus the energy output of level 1 is set to 15 minutes.
2. Various modes can be selected based on your need, press "Mode switch" to select the appropriate output mode which are Positive output, negative output, and mixed output mode.
3. Press the "Time increase/decrease button" which allows you to set the Working time from 1-30 minutes.
4. Press the "Start"/"Pause "button to start the operation. The power energy level is adjustable which allows being set from level 1 to 10.
5. When you want the Ultrasonic probe to stop working during the operation you can press the "Pause" button.
6. Press the "power switch" to turn off the equipment when the operation is end, and the "power switch" will display red at this time. Clean the Ultrasonic probe with 75% alcohol after unplugging the power plug, and store it in a dry place.

Recommended using time

Adjust the time according to the thickness of the skin, usually the treatment time does not exceed 15 minutes.

Warning: Do not put the metal part of the probe in contact with water

Warning

- Do not point the probe directly at the eyeball, touch the eye or massage the upper eyelid.
- It is forbidden for the following groups :
 - ① Pregnant women and those equipped with a cardiac chronometer;
 - ② Tumor patients and serious heart disease;
 - ③ Skin burns, scalds, sunburn, inflammation or other wounds;
 - ④ Allergy patients and people with contact dermatitis.
- It is forbidden to open the probe without cosmetic operation to prevent the risk of high temperature scald and reduce the service life of the probe due to overheating.
- Do not remove the stainless steel probe and handle by yourself. If you turn the stainless steel probe and handle, the probe may be damaged.
- This function shall not be used for other than cosmetic purposes.

Attention

- Before care, it is necessary to apply enough cream or essence on the care part to ensure that there is a good medium between the probe and the skin, so as not to pull the skin too much.
- When using, the probe should be close to the skin to prevent cavitation and damage to the skin.
- When used, the probe should not be in the output state without operation, in case of damage to the probe or reduce the service life of the probe.
- If the probe is too hot, it will burn the skin, so the use time should not exceed 15 minutes, sensitive skin should reduce the use time and energy intensity. Lengthening the use time will not enhance the nursing effect and may lead to danger. Attention should be paid to the care of a skin stay time should not exceed 5 seconds.
- After nursing, the skin will appear a short fever ruddy phenomenon, due to the vibration of the probe and heat production, increasing blood circulation, which belongs to the normal state.

Product Parameters

Product model	F-312B
Power	AC 100V-240V, 50/60Hz
Power Rating	11W
Product size	20.8×23×7.5 cm
Package size	37×27.5×14 cm
Package contents	Main Unit × 1
	Accessories Shelf × 2
	Stainless steel ultrasonic tip plane probe × 1
	Stainless steel ultrasonic plane probe × 1
	Power cord × 1

Troubleshooting

The Fault Phenomenon	Reason	Solution
The main machine does not work or crashes.	1. No power.	1. Check whether the fuse in the fuse bin in the power base is burned out. If burn out, replace the corresponding specifications of the new fuse. 2. Press the start button, the button will light up after power.
	2. The power cord is not properly inserted or connected.	2. Plug or connect the power cord properly.
Display screen doesn't work.	1. No power.	1. Turn on the power.
Accessory probe doesn't work.	1. Poor interface contact.	1. Unplug the output cable, and then reinsert the socket.
Weak output for Cold and hot hammer, ultrasonic, scrubber, RF, Galvanic and other accessories.	1. The tap position is not set well; 2. The high temperature of the accessories leads to low output power.	1. Adjust the high-grade position; 2. Appropriate use, do not open for a long time. Turn off the power switch when accessories are not in use.
Dermabrasion has weak suction.	1. The tap position is not set well; 2. The inside of the Dermabrasion blocked; 3. Aging of air tube; 4. The interface between the air tube and the suction pen is loose.	1. Adjust the gear position and adjust the air regulating valve; 2. Replace the Dermabrasion accessories; 3. Replace the new air tube and insert the interface.
Sprayer with weak spraying effect.	1. The tap position is not set well; 2. The water refill instrument is blocked inside; 3. Aging of air tube; 4. The interface between the air tube and the water refill instrument is loose; 5. The water in the bottle is insufficient.	1. Adjust the high-grade position; 2. Replace the accessories of the water refill instrument; 3. Replace the new air tube and insert the interface; 4. Pay attention to the content of water in the water bottle and add water in time.
The suction of the pen is weak.	1. The tap position is not set well; 2. The pen suction tube is blocked inside; 3. Aging of air tube; 4. The interface between the air tube and the suction pen is loose.	1. Adjust the high-grade position; 2. Replace the suction pen attachment; 3. Replace the new air tube and insert the interface.
The output of high frequency electrotherapy is weak.	1. The tap position is not set well; 2. Aging after long-term use of electric therapy rod.	1. Adjust the high-grade position; 2. Replace the electrotherapy accessories.
The host jitter.	1. Accessories are blocked.	1. Check whether the air tube of relevant accessories is blocked.

If problem is still not solved after above operations, please contact maintenance center or supplier to solve it.



Do not dispose of old or defective appliances in domestic garbage; this should only be done through public collection points.

- ▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.