

F-334 Multifunctional Beauty Instrument

INSTRUCTION MANU

Brief Introduction

F-334 is a multifunctional beauty instrument which has function such as Skin Scrubber and diamond dermabrasion. its indication is clear and easy to apply.

A : Connect all Join Parts to the Host Machine referring to different Join Parts User Instruction Manu.

B: Turn on the “on/Off” button to activate the LCD screen.

C : Press “FUNCTION” to select the desired function following order :1 Skin Scrubber, 2 diamond dermabrasion,

D: After turning to the desired function, set up working level and time, then press “START/PAUSE” button to start function.

FUNCTION

Skin Scrubber

Under the principle of Ultrasonic and Ion-introduction, the Skin Scrubber applies the technic of Ultrasonic through vibration to clean out blackhead, Hill diagnosed, Necrotic skin, residue from cosmetics, accumulated oil etc from skin surface in order to improve blood and lymph circulation in this area, strengthen cell’s transparency, improve system’s metabolic and cell reborn effect. There are 3 types of ultrasonic under this function, they are either Sine Wave(), Combination Wave () or Square Wave (which will be clearly shown on the LCD Screen. Ion-Introduction can improve skin’s inner environment, and help to remove spot, whiten skin and remove wrinkles.

USER INSTRUCTION


A. Connect Skin Scrubber and Lead Earth Wrist to the back of the Host Machine, Skin Scrubber should plug in to “SCRUBBER”, and Lead Earth Wrist to “LEAD EARTH”.

B. Turn on the Host Machine, and activate LCD Screen

C. Select the desired ultrasonic type, energy and time (Turn –On default : Continuous Wave, lowest level of energy, 15 Minutes), you can change setting anytime during working.

D. Press “START/PAUSE” button to start Skin Scrubber, put the LEAD EARTH on client’s wrist. Once there is circuit between the client and the scrubber, machine will start to work. The Scrubber sign on the screen will run when machine is working normally. Once there is separation between the scrubber and the client, machine will pause, and the Scrubber sign on the screen will stop running. During work, position the scrubber at a 45 °to the client treated area, and move forward gently and slowly which will have the cleansing effect to the skin.

PANEL INTRODUCTION

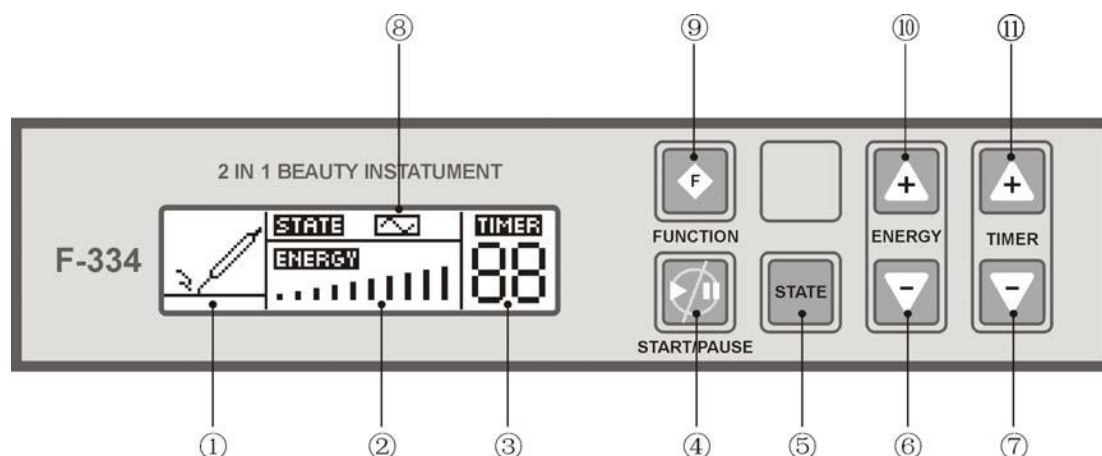

- | | |
|---------------------------------|---------------------------|
| ① The display of Scrubber | ② The display of Energy |
| ③ The display of timer | ④ “START/PAUSE” button |
| ⑤ No effect under this function | ⑥ “-” for Energy |
| ⑦ “-” for Timer | ⑧ Ultrasound Type Display |
| ⑨ Function Selection | ⑩ “+” For Energy |
| ⑪ “+” for Timer | |

NOTICE

- Under the situation when the Host Machine hoots every 5 seconds, and the Scrubber Sign on the screen twinkles, that means the Skin Scrubber and the Lead Earth do not have a circuit, this is caused by bad connection with the Host Machine. Check if the Scrubber is well plugged so as the LEAD EARTH Wrist, and restart Machine after all are checked.
- Check carefully if there is problem such as bad connection, damaged scrubber or Lead Earth, or no output from Host machine, if there is such problem, connect maintenance center, if parts need to be replaced, please replace them with producer's parts.
- During treatment, treatment area should be always moisture enough, if need to add nutrition, add those inviscid or diluted fluid but not those oily essence.
- Set up the desired Wave type, Energy and time before usage, avoid liquid on the host machine.

CAUTION

- Person under the following sickness is not suitable to use this instrument: Heart disease, Carrier of contagious disease, Neuroticism and all person under treatment with metallic material inside the body.
- Do not stay the Scrubber on one area for too long time in order not to give damage to skin.
- If the Skin lotion essence isn't sufficient enough on the surface of skin, add more.
- Pay attention on the angle between the scrubber and the treated area, working force is to be control during cleansing.
- Pay attention on the connection wire, extensive pulling is forbidden.
- Water or other corrosive liquid are forbidden from the Host Machine while working.
- deformed, damaged or polished scrubber is not allowed to use in order to avoid scrape on skin.
- If there is any abnormality during work, please turn off the machine and unplug. Contact sales representative and have expert to check it. Without permission, please do not open the cabinet.
- If the power wire is damaged, please replace it with the correct one. Wire can be purchased or exchanged from producer or the Exclusive After Sales Store.


Diamond Dermabrasion

1. Remove Cuticle, moisture skin, cleanse pores, lighten pimple pit and scar.
2. Absorb extra oil in pores and on the skin, give sheer to skin, and remove edema of face and eye area.
3. Increase skin's resilience, remove wrinkles, improve skin's quality and reveal skin's vitality.
4. Improve skin which has problem such as dark skin, rough skin, bad skin complexion, pigment and speck skin.
5. Lighten pigment, and stretch marks.
6. Remove cuticles from body's different area and become more moisture.

Vacuum User Instruction

For Blackhead: Manage to connect one side of the tube to the Vacuum plug of the Host Machine, and the other side connect with Diamond Dermabrasion sensors (9 sensors are provided for different level), then turn on the "POWER", switch function to Diamond Dermabrasion, set up the desired energy and time, then press "START/PAUSE" button to start working. The Vacuum Stick can be open in the middle, and cotton could be placed inside, while working, cotton should be placed inside to block the sucked substance to prevent damage to the Host Machine. And there is a small hole located at the front of the Vacuum Stick, put a finger to block the hole and treatment can start. While placing Glass Tube on treatment area, block or clear the hole to control vacuuming force. Do not let the tube stay long time on treatment area, it can cause skin damage. If there isn't any vacuuming force at the beginning, it is normal, wait a while and machine will begin working.

PANEL INTRODUCTION

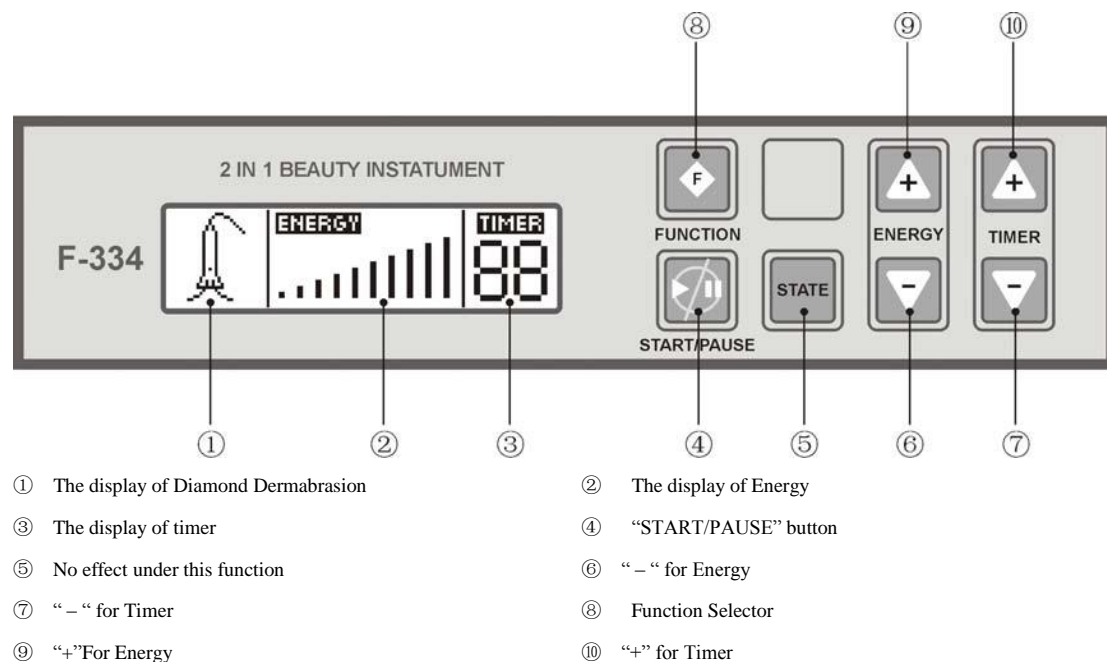

Basic Theory of Diamond Dermabrasion

The Diamond Dermabrasion instrument is working with mini diamonds of different thickness, with the control of working depth friction and Vacuuming energy to eliminate toxin from lymph, and help aging skin, pigment skin and big pore skin to become more tender and delicate, renew skin and help nutrition's absorption.

Instrument's characteristics

1. The newest Diamond Dermabrasion and skin moisture technology, safe and analgesia, without side effect.
2. There are 9 Diamond sensors, 3 for different areas of face, 2 for different eye area and remove wrinkles, 4 for different body area and bigger area.
3. Personalized design: Handle is designed to fit hand perfectly for beautician's convenience.
4. Easy manipulation with skin's best result, it is natural and safe.
5. NO need for any powder.
6. No need to rest after treatment, and make up can be put right after, no interruption to daily life.

CAUTION

1. Forbid to absorb any liquid into machine, or it will damage the air pump.
2. Do not over pull the silicone tube, which may break it.
3. Forbid to add lube oil to the air pump.

Functional Output

