

F-801C

Beauty Instrument

User Manual



Welcome to use our products. A0
Please read the user manual carefully before using the products.

About This Manual

This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, we will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of manufacturer.

Safety Guidance

**To ensure your correct and safe use of this product, please read all the instructions before use!
Please follow all safety precautions!**

	Warning!	<p>Indicates a potential dangerous situation that, if not avoided, could result in serious loss of life.</p> <ul style="list-style-type: none">• This product is only for beauty practitioners with professional qualifications.• Do not use or store the product in a damp place such as bathroom to avoid malfunction and danger.• Do not use this product near flammable items (e.g., alcohol, thinner, gasoline, nail polish remover, sprays, etc.).• Do not use this product under the following conditions: when the power cord or plug is damaged or gets hot; when the plug in the power socket comes loose.• Do not use wet hands to plug and unplug power cord connectors and use this product.• Please make sure that the working voltage of this product conforms to its rated voltage.• Before use, make sure that the power plug and the power cord connector are fully plugged in, so as not to cause danger.• When not using this product, please turn off the power and pull out the plug.• Please regularly clean the dust accumulated on the power plug and power cord connector to avoid failure of insulation caused by moisture, dust and other factors.• Be sure to use the special power cord assembly for this product. Do not use the power cord assembly on any other appliance.• Do not wrap the power cord around the product, otherwise it may distort and malfunction the power cord.• Keep out of reach of children. Do not allow children to use this product.• Please do not modify, disassemble or repair this product by yourself. If you need to repair it, please contact the manufacturer, agent or professional maintenance personnel to check and repair it, so as not to cause danger.• Please contact distributor, service center or related personal with appropriate skill and experience for repairing if the plug is damaged, in case of any danger.
	Attention!	<p>Represents a potentially dangerous situation which, if not avoided, can result in general injury to person.</p> <ul style="list-style-type: none">• Operators should pay special attention to the safety of personnel and equipment when using.• If the product is abnormal, please stop using it immediately and contact the manufacturer or dealer for relevant treatment.• This product and related accessories should be handled lightly.• Please properly handle the packaging bag of this product to avoid accidents such as asphyxiation of infants, and keep it out of reach of children.

Components




Main Unit




Power cord

◆ Rear Panel

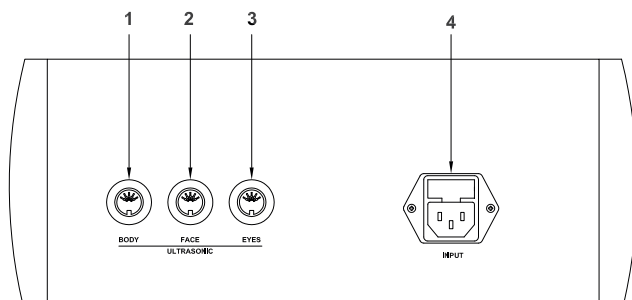

1. Body probe
2. Face probe
3. Eye probe
4. Power socket

◆ Front Panel

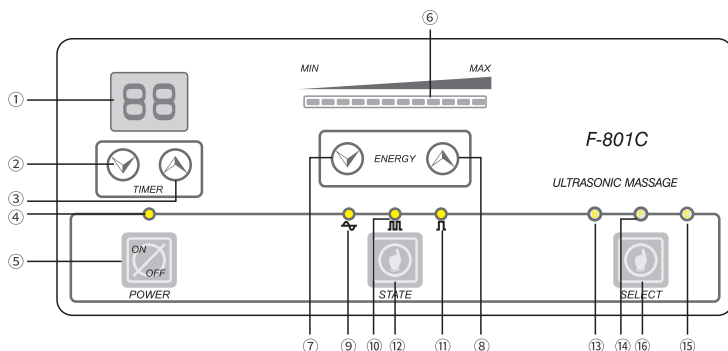

- ① TIMER: Countdown time, minute display, which is adjustable from 1~30 minutes.
- ② TIMER DECREMENT BUTTON: Short press minus 1, long press can reduce time continuously.
- ③ TIMER INCREMENT BUTTON: Short press plus 1, long press can increase time continuously.
- ④ "POWER" INDICATE LIGHT: When the light is on, the probe is working. And when the light is off, the probe stops.
- ⑤ ON/OFF BUTTON: The button is used for Power On/Off and Pause/Start operation. Short press for 1 time, the machine enters the stand-by state; Short press again can enter working state and be controlled via ON/OFF; and long press for 2 seconds, the machine is power off.
- ⑥ ENERGY DISPLAY: Total 10 levels, the energy increases when indicate light changing from green color to red color.
- ⑦ ENERGY DECREMENT BUTTON: Press once to lower one gear, lowest is gear 1.
- ⑧ ENERGY INCREMENT BUTTON: Press once to increase one gear, highest is gear 10.
- ⑨ "—" INDICATE LIGHT: Red light means continuously outputting.
- ⑩ "лл" INDICATE LIGHT: Yellow light means intermittent output.
- ⑪ "Л" INDICATE LIGHT: Green light means hybrid output.
- ⑫ "STATE" BUTTON: Short press each time to choose the output method. Different indicate light means different output method.
- ⑬ "B" INDICATE LIGHT: When the light is on, body probe is working. (Dedicated for body treatment)
- ⑭ "F" INDICATE LIGHT: When the light is on, face probe is working. (Dedicated for face treatment)
- ⑮ "E" INDICATE LIGHT: When the light is on, eye probe is working. (Dedicated for pouches and small area treatment)
- ⑯ "SELECT" BUTTON: Short press each time to choose the using probe. Different indicate light is corresponding to different probe.

Product Parameters

Product name	Beauty Instrument
Product model	F-801C
Product size	360×360×185(mm)
Net weight	4 kg
Package contents	Eye Probe × 1
	Face Probe × 1
	Body Probe × 1
	Main Unit × 1
	Power cord × 1
	Manual × 1
Rated voltage	AC 100-120V / AC 220-240V
Frequency	50/60Hz
Power	20W

◆ Operation Cautions



Warning!

- Do not touch the probe surface and output interface when the machine is powered on.
- Do not direct the probe to contact the eye area.
- Prohibited for use by:
 1. Allergy patients;
 2. Skin burns, scalds, sunburn, inflammation or other wounds.
- It is forbidden to turn on the probe when do not do treatment. This is to prevent the risk of scald due to high temperature and maintain the service life of the probe due to overheating.
- Do not remove the stainless steel probe and handle by yourself. If you turn the stainless steel probe and handle, the probe may be damaged.



Attention!

- The probe should be kept clean at all times and disinfected with 75% alcohol after each use.
- Do not stay longer than 5 seconds on one skin.
- Large plane probe is only suitable for the body, limbs and hips to lose weight; face probe is suitable for facial beauty, when doing facial beauty, do not put the adjustment of "ENERGY" exceeds 7 gears. Avoid using it for too long. If the operation time is too long, it will not only fatigue the skin and affect the absorption, but also raise the temperature of the probe due to the mechanical vibration generated by the output.

◆ Operation method

Care preparation

1. The customer is in a stable position and adopt a comfortable posture.
2. Ask the customer whether there are any contraindication symptoms.
3. Explain to the customer the operation to do the care content and matters needing attention.
4. Beauticians clean and disinfect their hands.
5. Help customers clean and toner their skin.
6. Test customer's skin: cold and heat sensitivity and skin sensitivity test.
7. Put the instrument in a stable place, check the plug, socket, wire and interface end is safe and away from water.
8. Press the main POWER switch "POWER" key, enter the standby state, output mode indicator red light, probe output "E" indicator. The light is on, the time window is displayed as "15" minutes, and the energy gear is "1".

Operation procedure

1. Adjust the "SELECT" key to SELECT the probe for proper care.
2. Adjust the STATE key and select an appropriate output mode.
3. Adjust the "ENERGY" button to select the appropriate output ENERGY (you can first use water to debug on the probe surface and determine the output gear).
4. Adjust the "TIMER" key to set the working time.
5. Press the "POWER" button, the "POWER" indicator lights up, the probe starts output work, and the beauty operation begins.
6. When not in use, short press the "POWER" button to suspend work; Or long press "POWER" for about 2 seconds to turn off the output And display.
7. After the host runs the preset time, it will sound "di", indicating that the work has been completed, and the "POWER" indicator is off. The system returns to the standby state.
8. after operation, pull out the power plug, wipe the host and accessories clean, keep in a cool, dry place.

Recommended using time

Under normal circumstances, a nursing time should not exceed 15 minutes.

Instrument Maintenance

- After the nursing operation, the probe should be sterilized with 75% alcohol and stored in a dry place.

Caution: Do not wet the metal part of the probe.

Troubleshooting

The Fault Phenomenon	Reason	Solution
The main machine does not work or crashes.	1. No power.	1. Turn on the power.
	2. The power cord is not properly inserted or connected.	2. Plug or connect the power cord properly.
The probe has no output.	1. No power.	1. Turn on the power.
	2. The power cord is not properly inserted or connected.	2. Plug or connect the power cord properly.
	3. Didn't adjust the power output gear.	3. Set the "ENERGY".
	4. Picked a wrong probe.	4. during operation, the light is B for body sensor output, F lights for facial probe, E lights for eye probe.
	5. Three probes are unable to work.	5. the power cord connection, check the host at the back of the power supply within the insurance whether insurance tube in tube warehouse has been burned. If already bad, should replace the new 250V / 1A insurance tube.

If problem is still not solved after above operations, please contact maintenance center or supplier to solve it.



Do not dispose of old or defective appliances in domestic garbage; this should only be done through public collection points.

▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.