

F-806

High frequency electrotherapy beauty instrument

User Manual

About This Manual

This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, we will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of manufacturer.

Safety Guidance

To ensure your correct and safe use of this product, please read all the instructions before use! Please follow all safety precautions!

	Warning!	Indicates a potentially dangerous situation that, if not avoided, could result in serious loss of life. • This product is only for beauty practitioners with professional qualifications. • Do not use or store the product in a damp place such as bathroom to avoid malfunction and danger. • Do not use this product near flammable items (e.g., alcohol, thinner, gasoline, nail polish remover, sprays, etc.). • Do not use this product under the following conditions: when the power cord or plug is damaged or gets hot; when the plug in the power socket comes loose. • Do not use wet hands to plug and unplug power cord connectors and use this product. • Please make sure that the working voltage of this product conforms to its rated voltage. • Before use, make sure that the power plug and the power cord connector are fully plugged in, so as not to cause danger. • When not using this product, please turn off the power and pull out the plug. • Please regularly clean the dust accumulated on the power plug and power cord connector to avoid failure of insulation caused by moisture, dust and other factors. • Be sure to use the special power cord assembly for this product. Do not use the power cord assembly on any other appliance. • Do not wrap the power cord around the product, otherwise it may distort and malfunction the power cord. • Keep out of reach of children. Do not allow children to use this product. • Please do not modify, disassemble or repair this product by yourself. If you need to repair it, please contact the manufacturer, agent or professional maintenance personnel to check and repair it, so as not to cause danger. • Please contact distributor, service center or related personal with appropriate skill and experience for repairing if the plug is damaged, in case of any danger.
	Attention!	Represents a potentially dangerous situation which, if not avoided, can result in general injury to person. • Operators should pay special attention to the safety of personnel and equipment when using. • If the product is abnormal, please stop using it immediately and contact the manufacturer or dealer for relevant treatment. • This product and related accessories should be handled lightly. • Please properly handle the packaging bag of this product to avoid accidents such as asphyxiation of infants, and keep it out of reach of children.
	Caution, electric shock!	During operation, please do not touch the junction of the high-frequency electrotherapy handle and the electrotherapy tube with your hands to avoid electric shock.

Product Introduction

Portable high frequency electrotherapy beauty instrument is through the ultra-high voltage low current to stimulate the skin, with antibacterial, promote metabolism, improve skin texture.

Scope of application


Suitable for beauty, massage, nursing.

Applicable symptoms


It is suitable for dry, seborheic and acne skin.

Components

The host of high frequency electrotherapy beauty instrument

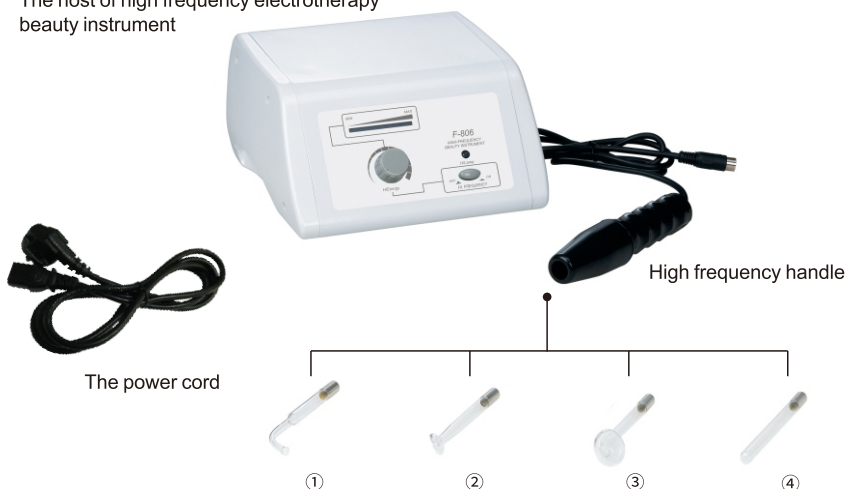

- ① Point-shaped electrode pipe : For the nest side area, such as the nose.
- ② Spoon-shaped electrode pipe : For the jaw and other middle area of the site.
- ③ Mushroom-shaped electrode pipe : For the cheek, forehead, neck and other parts of large area.
- ④ Straight rod galvanometer tube : Suitable for indirect nursing.

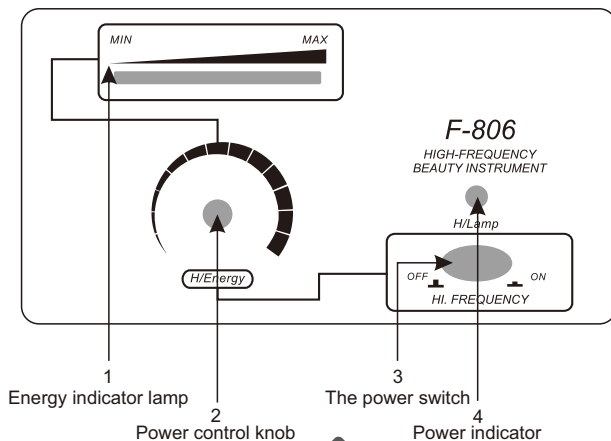

Product Parameters

Product name	High frequency electrotherapy beauty instrument
Product model	F-806
Product size	245.5×240×130 (mm)
Net weight	2.52kg
Package contents	The host of high frequency electrotherapy beauty instrument *1
	High frequency galvanometer handle *1
	Electrotherapy tube *4
	The power cord *1
	User manual*1
	Warranty card *1
Rated voltage	AC 100-120V/AC 220-240V
Frequency	50/60Hz
Power	10W

◆ Operation Cautions




Warning

- It is forbidden to touch the output interface of the main machine of the high-frequency electrotherapy beauty instrument or the output interface of the handle of the high-frequency electrotherapy when the main machine is powered on.
- Do not point the tube directly or in contact with the eyes area.
- This product is prohibited for the following people and parts:
 - ① Pregnant women and those equipped with adjusters (e.g. cardiac adjusters).
 - ② People with vascular diseases and high skin sensitivity.
 - ③ Having metal implants or metal dentures embedded in the body.
 - ④ People with skin wounds and other conditions.
 - ⑤ Body parts with thick hair.
- Do not use skin lotions containing alcohol, as alcohol is highly flammable. If alcohol-based toner is required, only apply it after using this product.
- Do not grasp the junction between the high-frequency handle and the electrode tube by hand during operation to avoid being shocked.
- The skin retention time of each part should not exceed 5 seconds.



Attention

- Before use, check whether the electrode tube or the metal head on the tube is loose and whether the electrode tube cracks or is damaged. If so, do not use it and replace the electrode tube immediately.
- The current intensity must be gradually reduced to zero and the power switch must be turned off before the electrode tube can be inserted or pulled out.
- When inserting or pulling out the electrode tube, first wear rubber gloves and hold the electrode tube with thumb and forefinger: insert the tube to the bottom of the hole until it is firm, and do not push the tube with the palm to prevent the glass from breaking and causing injuries; It is forbidden to pull out vigorously, so as to prevent the metal head on the electrode tube from falling off or breaking, which may cause injury.

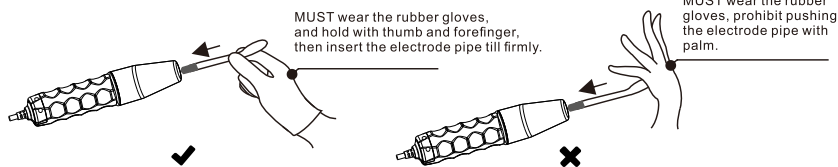

- When in use, the high voltage current output from the electrotherapy tube will cause interference. It is forbidden to approach the host machine and other electrical appliances of the high-frequency electrotherapy beauty instrument. If it interferes with the host, it may cause the host to not work or crack.
- When the electrode tube is less than 2.5cm from the skin but not in contact with the skin, the current intensity should be reduced immediately if sparks emit to irritate the skin. If the customer still feels the spark after the above operation, a small piece of dry cotton towel (nylon or other synthetic fiber towels are not allowed) put on the customer's face.
- When using, always keep the electrode tube on the customer's skin. High-frequency indirect care: when turning on and off the instrument, one hand must always be in contact with the customer's skin, never leave the customer's skin. Both sides feel the current and it causes discomfort.
- After use, the power must be turned off first, the electrode tube should be pulled out from the high-frequency handle and after disinfection and drying put it into the accessory box.


◆ Direct Care of High Frequency

Care preparation

- ① Put the customer in a stable position and take a comfortable posture.
- ② Check if the customer takes off all metal ornaments or not.
- ③ Check if the customer have any contraindications or not.
- ④ Explain the operation to the customer.
- ⑤ Cleansing and toning.
- ⑥ Skin testing.
- ⑦ Place the instrument in a stable place, check if the plug heads, sockets, wires and interfaces are safe and away from the water source.
- ⑧ Confirm that the electrode energy knob has been set to zero and sterilize the electrode pipe.

Operation procedure

- ① Select the appropriate electrode pipe and apply talc powder or ozone cream to help the pipe move better.
- ② The probe of the electrotherapy tube is placed in the customer's care part, press the "Hi.Frequency" key, open the switch, adjust the electrotherapy Energy "H/Energy" knob, gradually strengthen the Energy intensity to the customer's tolerance range. Be careful to turn the current down in the bone area. The free hand of the beautician should be placed on a pillow or hold the wire of the high frequency electric therapy handle without touching the customer.
- ③ When the customer is doing facial care, circle the skin like "之" , according to the sequence:

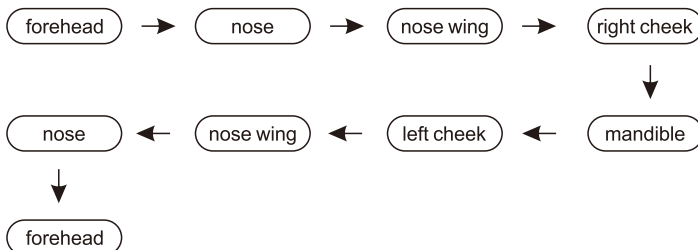

Please note that do not leave the electrode pipe on the skin. The lighter and shallower the movement, the greater the stimulation

- ④ Before the electric therapy tube is removed from the customer's skin, the "H/Energy" knob of electric therapy Energy is gradually reduced to counterclockwise, until zero, and then press the "Hi.Frequency" button to close the switch
- ⑤ Wipe the skin and perform toning
- ⑥ Clean the electrode pipe with boiling water and cleaning fluid, then dry with disinfectant wipes.

Note: Don't soak the metal part of the electrode pipe.

Recommended using time

- ① Oily skin: recommended 10-15 minutes.
- ② Dry, dehydrated skin: recommended 3-5 minutes.

◆ High-frequency Indirect Care

Care preparation

Care preparation is the same as direct care.
Please take the preparation of direct care for reference.

Operation procedure

- ① Wipe the electrode pipe with disinfectant wipes, and ask the customer to hold the pipe firmly. If necessary, apply talc powder on their hands to absorb the sweat of hands.
- ② Beautician puts one hand on the treatment location, and massages it with small circle.
- ③ In the range of customer tolerance, slowly adjust the electric therapy Energy "H/Energy" knob, gradually increase the current intensity.
- ④ Put the other hand on treatment, and massage gently with both hands. Fingers can feel the generated heat during massage.
- ⑤ Must remove hands from treatment part in the same time, so as to avoid electric shock.
- ⑥ At the end of the operation, the beautician will shift a hand from the customer's nursing department, first the electric therapy Energy "H/Energy" knob gradually to counterclockwise small, until zero, and then press the "HI.Frequency" key, off the switch.
- ⑦ Wipe the skin and perform toning.
- ⑧ Clean the electrode pipe with boiling water and cleaning fluid, then dry with disinfectant wipes.

Note: Don't soak the metal part of electrode pipe.

Recommended using time

Depending on skin types, response and tolerance, 8-20 minutes is generally recommended.

Instrument Maintenance

- When the electrode tube needs to be maintained, pull the tube out of the high-frequency handle first.
- The tube should be kept clean and dry. After use, the tube should be disinfected with 75% alcohol and stored properly.

Caution: Do not wet the metal part of the electrode tube.

Abnormal phenomena	Cause	Solution
The electric tube does not emit light, or it does not produce arc.	1. No power.	1. Turn on the power switch.
	2. The power cord is not properly inserted or connected.	2. Plug or connect the power cord properly.
	3. The electrode tube is not securely inserted.	3. Insert the electric therapy tube into the bottom of the hole until it is stable.
	4. The metal head on the electrode tube is loose.	4. Replace the electrode tube.
	5. The electrode tube is cracked or damaged.	
The main machine does not work or crashes.	1. No power.	1. Turn on the power.
	2. The power cord is not properly inserted or connected.	2. Plug or connect the power cord properly.
	3. The main machine is disturbed by high frequency current.	3. Keep the therapy tube away from the main machine and turn it off and turn on again.
High-frequency electrode has no output.	1. The high-frequency handle socket is not well connected.	1. Properly connect the socket of the high-frequency handle.
	2. The power voltage does not meet the requirements of the product.	2. Ensure that the working voltage conforming to the operating voltage of the product.
	3. High-frequency handle is damaged.	3. Replace the high-frequency handle.
	4. Electrode tube is damaged.	4. Replace the electrode tube.

If problem is still not solved after above operations, please contact maintenance center or supplier to solve it.



Do not dispose of old or defective appliances in garbage; this should only be done through public collection sites.

▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.