

F-808

Beauty Instrument

User Manual




Welcome to use this product! A0
Please read this instruction carefully before use.

About This Manual

This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, SILVERFOX will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of our company SILVERFOX.


Safety Guidance

To ensure your correct and safe use of this product, please read all the instructions before use! Please follow all safety precautions!



	Warning!	<p>Indicates a potentially dangerous situation that, if not avoided, could result in serious loss of life.</p> <ul style="list-style-type: none">• This product is only for beauty practitioners with professional qualifications.• Do not use or store the product in a damp place such as bathroom to avoid malfunction and danger.• Do not use this product near flammable items (e.g., alcohol, thinner, gasoline, nail polish remover, sprays, etc.).• Do not use this product under the following conditions: when the power cord or plug is damaged or gets hot; when the plug in the power socket comes loose.• Do not use wet hands to plug and unplug power cord connectors and use this product.• Please make sure that the working voltage of this product conforms to its rated voltage.• Before use, make sure that the power plug and the power cord connector are fully plugged in, so as not to cause danger.• When not using this product, please turn off the power and pull out the plug.• Please regularly clean the dust accumulated on the power plug and power cord connector to avoid failure of insulation caused by moisture, dust and other factors.• Be sure to use the special power cord assembly for this product. Do not use the power cord assembly on any other appliance.• Do not wrap the power cord around the product, otherwise it may distort and malfunction the power cord.• Keep out of reach of children. Do not allow children to use this product.• Please do not modify, disassemble or repair this product by yourself. If you need to repair it, please contact the manufacturer, agent or professional maintenance personnel to check and repair it, so as not to cause danger.• Please contact distributor, service center or related personal with appropriate skill and experience for repairing if the plug is damaged, in case of any danger.
	Attention!	<p>Represents a potentially dangerous situation which, if not avoided, can result in general injury to person.</p> <ul style="list-style-type: none">• Operators should pay special attention to the safety of personnel and equipment when using.• If the product is abnormal, please stop using it immediately and contact the manufacturer or dealer for relevant treatment.• This product and related accessories should be handled lightly.• Please properly handle the packaging bag of this product to avoid accidents such as asphyxiation of infants, and keep it out of reach of children.
	Caution, electric shock!	<p>During operation, please do not touch the junction of the high-frequency electrotherapy handle and the electrotherapy tube with your hands to avoid electric shock.</p>

PACKAGE CONTENTS

- These components are subject to change without prior notice.
- Appearance of product may differ from the image.

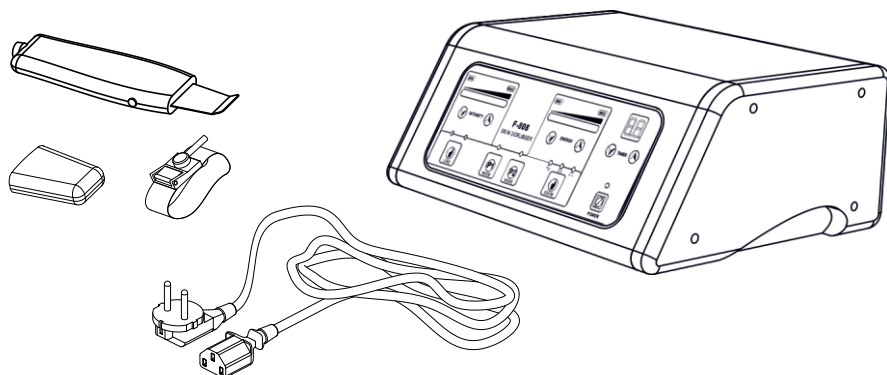

FRONT PANEL

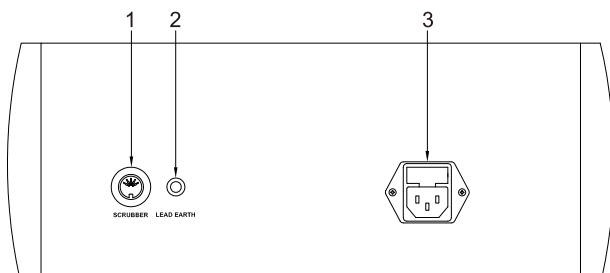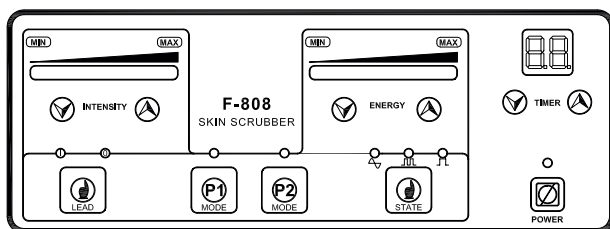

1. Skin scrubber probe
2. Bracelet
3. Power line input

INSTRUCTIONS OF USE

- Connect the power supply via the rear panel, indicated in the sketch as “POWER LINE INPUT”.
- Connect the accessories properly to the front/rear panel as indicated in the sketch.
- Turn on the “POWER” button.

GALVANIC USE

- Apply the correctly polarized product over the skin area to be treated.
- Press the “P1 MODE” button to activate the galvanic function.
- Select the polarity of the required current by pressing the “LEAD” button to switch between “+” (indicated by the “I” LED) or “-” (indicated by the “O” LED).

Positive pole (I)	Negative pole (O)
Acid pH solutions	Alkaline pH solutions

- Adjust the “INTENSITY” to the desired level.
- Use the probe over the skin.

ULTRASONIC PEELING USE

- Place the bracelet around the customer’s wrist.
- Apply the appropriate conductive product generously to the surface of the skin.
- Press the “P2 MODE” button to activate the peeling function.
- Select the desired kind of wave (continuous, pulse or combined) by pressing “STATE”.
- Adjust the “ENERGY” output to the desired level.
- Use the probe over the skin.

COMBINED GALVANIC & ULTRASONIC USE

- Place the bracelet around the customer’s wrist.
- Apply conductive gel to the treatment area.
- Press the “P1 MODE” and “P2 MODE” buttons to activate the combined function.
- Select the desired kind of wave (continuous, pulse or combined) by pressing “STATE”.
- Adjust both the “INTENSITY” and “ENERGY” to the desired output levels.
- Use the probe over the skin.

NOTES

- A positive earth control system is in place while the machine is operating. This means that unless the machine can find an earth return, the system will stop the current output to the probe. However, once the probe comes back into contact with the customer (the earth circuit is found), the system will restart the output immediately. This is a safety feature and should not be mistaken for a system fault.
- The probe will not vibrate when used on P1 program.

CAUTIONS

- People suffering from the following diseases are required not to use it: heart diseases, infectious diseases, people with metal implants and hypersensitive or vascular skins.
- Do not use the probe in one position on the skin for too long, to avoiding the risk of burning.
- Always handle the probe with care as it contains delicate components. Take particular care of the probe blade as any damage will cause scratching of the skins surface during use. Always store the probe safely when not in use.

FAQ

The probe suddenly stops emitting:

- If the probe is separated from the skin, it will stop emitting after 5 seconds. This is a safety feature and should not be mistaken for a system fault.
- Touch the skin with the probe and it will start emitting again after 5 seconds.

MAINTENANCE

- Any oil on the probe must be removed immediately.
- After use, please clean the probe and keep it. Store it in the probe-protecting sleeve at all time.
- Store the probe carefully to avoid damaging it.

TECHNICAL SHEET

Weights and sizes	Packing size (cm)	39×36×18.5
	Gross weight (kg)	4.3
	Product size (cm)	33×17/24×13
Working characteristics	Working frequency	25 - 27 KHz
	Preset working time	15 min
Electrical characteristics	Rated voltage	AC 220V - 240
	Frequency	50/60Hz
	Rated power	10 W



Do not dispose of old or defective appliances in domestic garbage; this should only be done through public collection points.

- ▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.