

# **USER'S MANUAL**

## **FOR ROTARY BRUSH INSTRUMENT (F-830)**

### **A. Brief Introduction on the Function**

It is the scarfskin at the surface of our skin that consists of stratum of keratose, hyalogen and graininess. The keratose stratum is the most external one, which is assembled with scalelike cells. The scalelike cells develop and die continually along with skin's metabolism. The dead cells would form a layer of velum, which block skin to breathe oxygen so that skin look dim and bring fleck indeed. ROTARY BRUSH can wipe off the dead surface-cells or accumulated dust, in pure mechanical way of the brush's running, in order to accelerate livingness and keep health to our skin.

### **B. Method and Steps for ROTARY BRUSH**

1. Plugging the connector of brush handgrip to the port back of the host, and choosing appropriate brush head to agree with client's skin type. In general, delicate or dry skin need downy brush head, on the other hand the harder ones suit crude or oil skin.
2. Switching the instrument's power on, then turning the 'Rotary Brush' switch to 'ON', in this time the lamp above the 'Rotary Brush' switch illuminate and the instrument is in the state of standby.
3. Holding the brush handgrip, adjusting the 'R/Energy' knob deasil, the function begin to run and the brush handgrip begin to circumvolve. The rotate speed will increase in synchronization of the increasing of adjusting angle. In operating, it is important to select a seemly rotate speed according to the skin type, skin sensitivity and cosiness of client's.
4. After operating over, immersing the brush head in clear water with a little cleanser for 5 minutes, then after clearout, immersing it in alcohol for half hour.
5. The brush head must be disinfected in ultraviolet radiation, and then be preserved in axenic chest.

### **C. Pretreatment before Operating**

1. Scrubbing the surface of object skin with waterish tampon.
2. Using STEAMER (or hot washrag) to intenerate the surface-skin in object area.
3. After finishing maintenance of clearing skin, it is proper to daub some nutrimental oil on the surface of object skin.

### **D. CAUTIONS**

1. When the brush head is running, it is forbidden to press the brush head excessively.
2. Don't to operate this instrument with any evident acidic or alkaline substance.
3. After operating over, please switch the power off and pull out the cord set.
4. Any client who is suffering hypersusceptible skin or dermatosis must not be used with this function.
5. If the capillary vessel in skin were injured, don't to clear skin with this function.
6. After operating over every time, the brush head must be cleanout and sterilize immediately.