
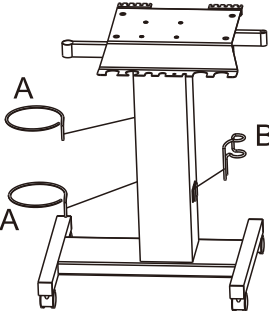
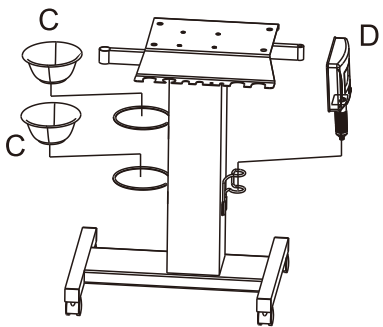
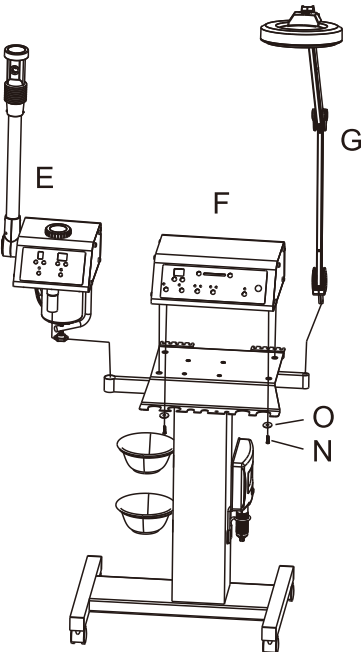
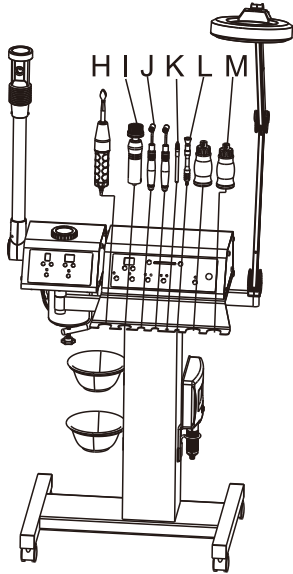
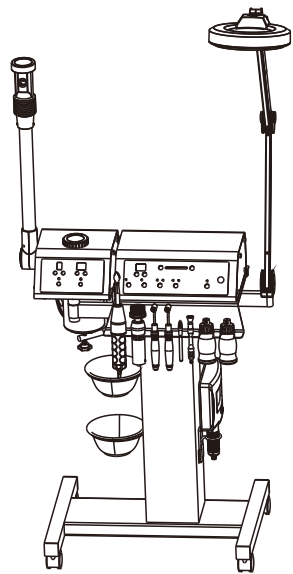


NO.	APPELLATION	PATTERN	QTY	ASSEMBLAGE		
A	Bowl holder		2			
B	Wood's lamp holder		1			
C	Mask bowl		2			
D	Wood's lamp		1			
E	Facial steamer		1			
F	Beauty instrument		1			
G	Magnifying lamp		1			
H	High frequency handset		1			
I	Brush handset		1			
J	Bio micro-current input & output handset		2			
K	Grounding bar		1			
L	Suction bar		1			
M	Spray bottle		2			
N	M4*20 Screw		4			
O	M4 steel washer		4			

F-800 Facial steamer

User Manual

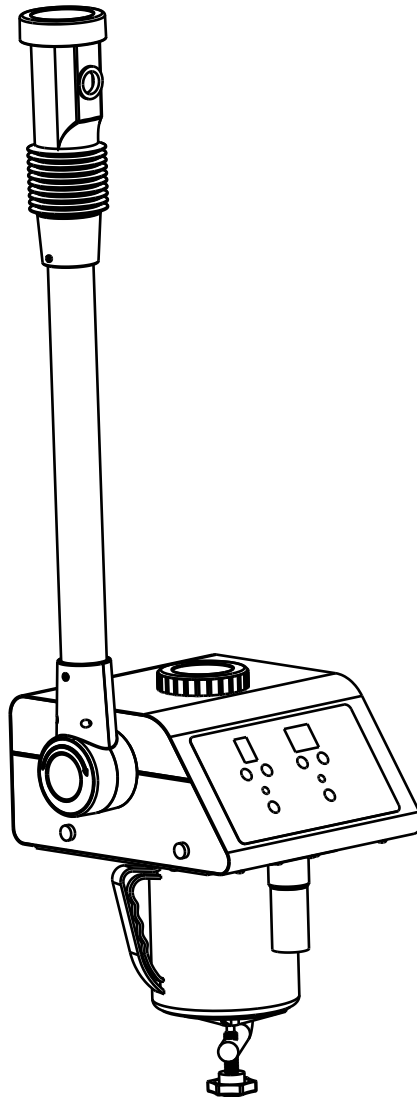


Welcome to use our F-800 Facial steamer,
Before use the bed, please read the user manual carefully, thanks.

Warning

1. Never leave facial steamer unattended when power on.
2. This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
3. It can only be used accordance with the inherent use described in the instruction manual, and no accessories applied without recommendation by manufacturer.
4. When the power cord or plug is damaged, when the facial steamer is abnormal or overturned, damaged, or the machine ship water, do not continue to use the machine, should be returned to the service center for inspection and maintenance.
5. Keep the power away from high temperature, and do not wrap the power cord around the facial steamer. Do not work under direct sunlight for a long time, or it may cause a variety of unstable conditions of the machine.
6. It is forbidden to use facial steamer in sleeping.
7. Forbid to insert or drop anything into facial steamer.
8. Only be used indoor, not outdoor.
9. Facial steamer
10. When fill water to steamer, shall pull out the plug. The steamer can fill water only and shall not exceed the limit.
11. Extended power cord should not be used.
12. Steamer should be use together with trolley or stand.
13. Forbid to spray vapor to same place for long, or directly down to the client's face too closely, otherwise, it will cause scald.
14. When fill water to steamer, pour water and clean the steamer, shall pull out the plug. Confirm steamer cooled before clean and store it. Move steamer carefully when hot water in bottle.
15. Do not touch high temperature part.
16. Forbid to use the steamer under humid environment, like bathroom, etc.
17. Forbid to open up the machine by oneself. If there is anything unusual, please consult the dealer, and send to professional technicians for inspection.
18. Forbid to modify heating coils and other parts of steamer.
19. When fill in water, please take out herb bag to avoid water were contaminated. (Suitable to herb filter net).
20. When it's heating or 1 minute after heating, forbid to open the lid to avoid scald.

Function


1. Intenerate the necrotic skin cells for cleaning in the subsequent treatment.
2. Open up the skin pores for deeper cleaning.
3. Penetrate into pores to soften and clean the grease, back head, make-up residues and dirt.
4. Help skin pore release toxin.
5. Soften wrinkle temporarily.
6. Activate the blood circulation, moisten the skin cell, improve metabolism and renew the skin.
7. Help to treat acne, stains, whiten and nourish skin by using appropriate herbs.

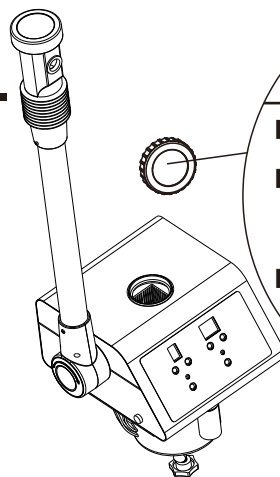

Warning


- Can fill in clean water, forbid to fill in essential oil and herb etc.
- When fill in water, should take out herb bag to avoid water were contaminated. (Suitable to herb filter net).
- When it's heating or 1 minute after heating, forbid to open the herb lid to avoid scald.

OPERATION INSTRUCTION

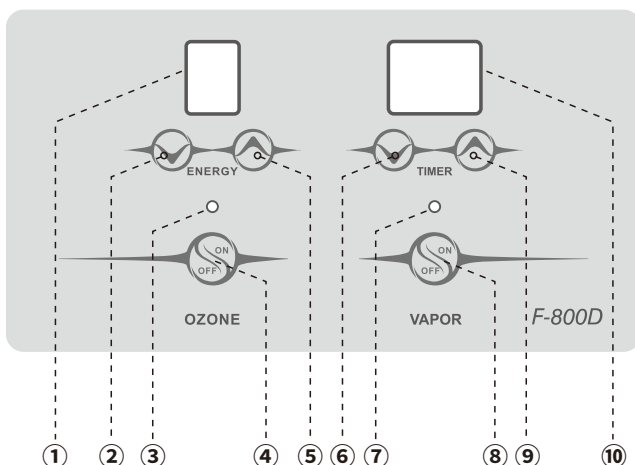


- | | |
|-----------------------------|----------------------------|
| ① Energy indicator light | ② Energy decreasing button |
| ③ Ozone indicator light | ④ Ozone ON/OFF button. |
| ⑤ Energy increasing button. | ⑥ Time decreasing button |
| ⑦ Spray indicator light | ⑧ Spray ON/OFF button |
| ⑨ Time increasing button | ⑩ Timer |


1. Place the water cup upon the water cup tray and then tighten the screw. Next, take the herbs cover lid up and pour the distilled water into the water cup. The water level should be between the MAX and MIN limit of the cup.

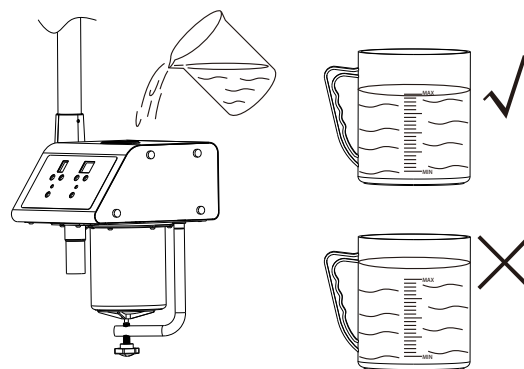

2. For herbs treatment, please take out the filter mesh, put the herbs bag into the small steel cup, then place the filter mesh and cover lid back. Please see in the following picture No.3.

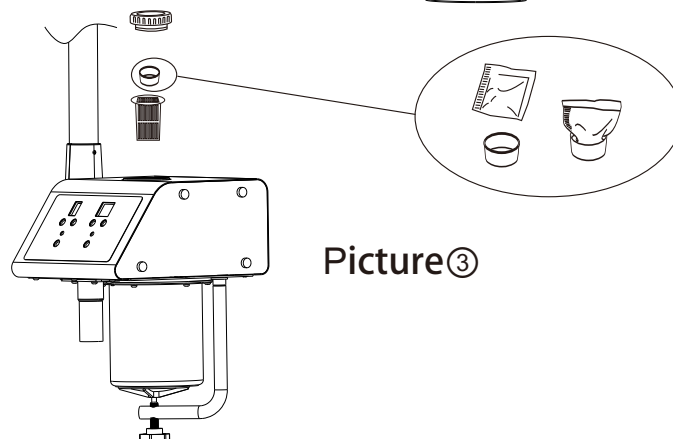

3. For fragrance treatment, please drop some essential oil in the balsam case in the spray head. The spray head can be adjusted at an appropriate angle. Please picture No.4 below.

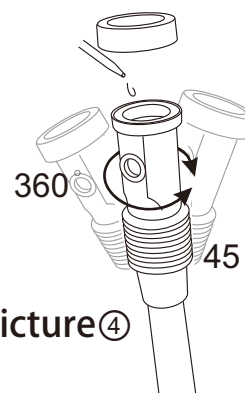

4. Connect the power plug.
5. Automatic inspection function after opening the steamer.

Press "VAPOR", the green indication light on the left will be on. The timer will indicate "15" minutes. Energy will indicate "0". The mainframe can check the water level in the bottle automatically. If the water level is between the MAX and MIN level, then the mainframe will standby. If the water level is lower than the MIN level, the mainframe will beep, and lock all buttons except for the "VAPOR" button.

6. Five level spray output

Press "+" or "-" keys under "Energy" to adjust the spray output from "0-5". "5" is the biggest output. Before the vapor sprays out of the pipe, we suggest to choose "5" first, then adjust the output level according to different requirements.

7. Time setting

Press "+" or "-" keys under "TIMER" to set time which range is from "0-99" minutes. Default time is "15" minutes once the instrument's power is on.

8. The function of Ozone to kill the bacteria.

Switch on the "OZONE" key. --Green lamp on the top is on. The Ozone lamp is working. There are obvious ozone smell in the vapor along with distinct atomization.


9、 During operation

when the water level drops to minimum limit, the mainframe will automatically response immediately and turn off the heater, lock up all the keys at the same time, except for the "VAPOR" key and accompany with alarming by constant buzzing sound. Besides, it won't work until the water is added to the normal level, press the button to proceed the operation.

10、 After the preset working time is up, the mainframe will switch off "VAPOR" key automatically and buzz 5 times, and the preset time resumes to 15 min. The mainframe stands by and won't work under this status.

Cautions

1、 When fill in water to bottle, avoid water overflowing from steamer shell. Otherwise, it probable cause electric leakage. Take out the herb filter net and lid, fill in purified water to bottle. The water level should between the "MAX" and "MIN" level limit.

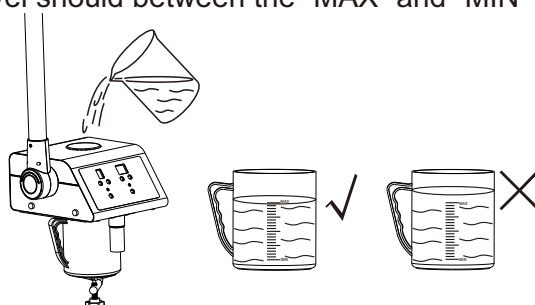

2、 If air or water leakage occurs during working, first switch off the "POWER" button immediately, and cut off power, then take off water cup and check whether the rubber gasket is properly fixed, reinstall it. If nothing improved, please consult dealers.

3、 When operating the machine, nozzle must not aim at client's face until the vapor sprays out thinly and evenly, keep a distance of 40cm from the face. Vapor should spray firstly from client's jaw, then all over the face.

4、 Since the ozone can kill the skin bacteria, it is not good to use for long time. Otherwise, it will harm to skin and decompose herbs' nutrition. The period of using time should be based on different types of skin.

5、 The machine should be well grounded.

6、 When add water to bottle, must take out herb bag before filling. Otherwise, it will cause scald by spraying hot water. The herb bags should be replaced at most 2 times using.

Choose different spray distance and times base on various Skin Condition

Skin Condition	Distance	Time
Greasy skin, poor blood circulation	30-35cm	15min
Normal skin	30-35cm	10min
Dry Skin	30-35cm	5min
Sensitive skin	35cm	5min
Pigmented skin	30-35cm	10min

Maintenance

1、 Clean the scale off from the water cup and the heater elements at least one time per week. (Please use water that contains 7% acetic acid or citric acid to clean the scale)

2、 Keep eyes on the water level during working.

3、 Inspect rubber gasket of bottle each 2 weeks to confirm if the gasket is deformation. If yes, please replace it on time.

Repair Instruction

1. If steamer spray water, its main reason is there are impurity in mixed in water. Clean the bottle and heating tube by table vinegar. Then wash them with clean water. If nothing changes, please send back to dealer for inspection and repairing.
2. If heating tube appear scale after use tap water for period. Please try to clean it with table vinegar if it's hard to clean it by water.
3. If spray fewer fog, please check whether have turned on "OZONE" button. If nothing changes after turned on and turned off) the "OZONE", it should be quartz burner stop working. Please contact dealer ask professional technicians for inspection and repairing.

Parameter

Item No.:	F-800
Voltage:	AC 100-120V~/220-240V~
Frequency	50/60Hz
Power	650W/800W
Net Weight	4.4 kg

Content of harmful substance in Products

Parts Name	Harmful substance					
	Pb	Hg	Cb	Cr(VI)	PBB	PBDE
Host	×	○	○	○	○	○
Bottle	○	○	○	○	○	○
Water tank	○	○	○	○	○	○
Stool	○	○	○	○	○	○
Accessories	○	○	○	○	○	○

This sheet is made according related stipulation of SJ/T 11364.

- :Means content of this harmful substance is below the standard of GB/T 26572 stipulation.
 ×:means the content of this harmful substance is over the standard of GB/T 26572 stipulation.

The steamer accord with RoHS environment protection: Only the electronic building brick of printed circuit module include Pb. And it cannot be replaced base on current technology.

The period of Environment protection means the harmful substance which content in the steamer will not be divulged in normal using. It won't contaminate the environment or harmful to human body when use it.



Special Declaration



- 1、 Must turn off the power switch and pull out power plug if steamer without water or lack of water. Not allow to connect power when water level is abnormal. Otherwise, user should be responsible for its accident if not operate according. Please reach manual carefully before using or operate it among the guide from professional beauticians.
- 2、 Forbid using to children. Children are not allowed to play around when steamer is working or within 1 hour after standby.
- 3、 The steamer is working by high temperature of heater. To prevent accident or fire which caused by steamer abnormal working, please do not leave facial steamer unattended when power on.



Do not dispose of old or defective appliances in domestic garbage;
this should only be done through public collection points.

Please check and verify products specifications before using. This user manual is only used for reference.
The product specifications will be subject to change or update without further notice.

F-821

Five in one instrument

Operating Manual



Please read the instructions carefully before using the products

1. Warning

1. Cut off the power supply when maintaining and cleaning the machine.
2. Unless being monitored or instructed by personnel who can be responsible for their personal safety, the people who have physical obstacles, tactile disorders, mental disorders, lacking of experience, common sense of children, are not allow to use the equipment.
3. Never leave machine unattended when power on.
4. In case of any abnormality, please contact the dealer and call professional maintenance personnel for inspection. Unauthorized disassembly of the main engine and any accessories thereof is prohibited.
5. It can only be used accordance with the inherent use described in the instruction manual, and no accessories applied without recommendation by manufacturer.
6. When the power cord or plug is damaged, when the machine is abnormal or overturned, damaged, or the machine ship water, do not continue to use the machine, should be returned to the service center for inspection and maintenance.
7. Keep the power away from high temperature, and do not wrap the power cord around the machine body. Do not work under direct sunlight for a long time, or it may cause a variety of unstable conditions of the machine.
8. It is forbidden to put the machine in the environment with high humidity.
9. Extended power cord should not be used.
10. This product is only available for professionals with qualifications.
11. Clients are not allowed to reach any metal objects like jewelry, if wear any, must take off all objects before treatment.
12. That pregnant women and people fitting rhyme adjuster by operating (like heart-rhymeadjuster) are not allowed to use Hi-frequency electrotherapy.
13. Cream or mask used in Hi-frequency electrotherapy treatment must not contain spirit. If have to use any make-up water with spirit, may use it after treatment.
14. Do not touch eyeball with any electrode pipe.
15. Never load or unload the glass electrode pipe, never touch the main output interface or glass electrode pipe interface of the high frequency handset when power on. Customers who are sensitive to skin, injured, inflamed, or telangiectasia, as well as pregnant women are not allowed to use Galvanic.

2. Introduction.

This instrument including five functions: High-frequency, Vacuum & Spray, Galvanic, Rotary brush. Following are introduced separately.

1. High-frequency

1) Theory of High Frequency Electrotherapy

High Frequency Electrotherapy instrument generates high speed vibrating current, which may convert to heat energy and kill bacteria. As the vibration is too fast, it won't cause muscle shrinkage. And it gives physical stimulation.

- a. Accelerate the blood circulation.
- b. Stimulate the glandular activity.
- c. Help the skin for excretion and absorption.
- d. Promote metabolism.
- e. Kill bacteria.
- f. Generate heat energy inside skin tissue.
- g. Help beauty cream penetrate into deep skin.

2)Usage

First choose a proper glass electrode pipe and insert to the output mouthpiece of high frequency handset, then connect to the instrument.

Glass electrode pipes assignment position:

- a. Mushroom-shape electrode pipe—used for large area of skin: face, forehead, neck, etc..
- b. Ladle-shape electrode pipe—used for medium area of skin: chin, etc.
- c. Dot-like electrode pipe—used for small area of skin: nose, etc..
- d. Straight electrode pipe—used for indirect treatment.

3)Direct usages:

A: Beautician grasps the handle of hi-frequency electrotherapeutic pole(try not to touch glass electrode pipe with hands or metal objects), and turn the button to minimum.

B: then touch the body part that is to be treated with glass electrode pipe, power on “START/PAUSE button” to start work, slowly adjust the button clockwise until there is a gentle output. If the face is covered with cream or mask, move the glass electrode pipe upward from clients neck, slowly slide to chin, face, cheek, nose and forehead. In the course of treatment, you should slowly adjust the button to the appropriate intensity. As the skin tissue generates heat energy, it may comfort and calm the skin by directly using the Hi-frequency electrotherapy, meanwhile it is efficiency fort killing bacteria. It is applicable to oily and with acne skin.

C: After treatment, slowly twist the button to minimum, power on “START PAUSE button” to stop work, and take away the glass electrode pipe from the client's face.

4)Indirect usages:

First let the client hold the straight electrode pipe, and adjust the button to the minimum, power on “START/PAUSE button” to start work, gradually turn the button to the desired intensity, then the beautician massages client's treated part. In the whole process, the beautician does not touch the electrode. The beautician must power off before taking away the glass electrode pole from the client. Indirect usages will accelerate and comfort the skin at the same time.

2.Vacuum & Spay

Vacuum—Use the small glass vacuum pipe to vacuum the black-head, white-head, acne, blain, dirt and grease etc..

Spray—Use the spray bottle to spray the make-up water, antipyretic or shrink lotion to diminish inflammation, comfort and nourish the skin.

1)Vacuum

Connecting the single air pipe to the “Vacuum” pipe marks nozzle, the other head to the stick assembly. Three small glass vacuum pipes for option to perform black-head vacuuming. It is advisable to stuff some absorbent cotton into the glass vacuum pipe for stopping dirt going into the pipe so as prevent damaging the instrument. The glass vacuum pipe must not stay on the face too long so that it will scald the face. It is normal if there is no vacuum at the beginning, it will resume to normal condition after a while.

2)Spay

Connecting the single air pipe to the “SPARAY” pipe marks nozzle, the other head to the small spray bottle. Two spray bottles that can contain different liquid, it will spray out by pressing the nozzle on top of spray bottle with finger, spray downward from forehead during working,

prevent liquid going into the nose, ask client to close eyes and shut mouth. Keep a adequate distance from the face.

3. Galvanic

1) Theory:

By mean of movements of DC positive and negative ions, the galvanic current treatment is helpful for leading in nutrition and leading out impurity from skin. The nutrition leaded in is to penetrate the water-soluble nutrition into the deep skin, directly supplying nutrition for skin. Impurity leaded out is to import the alkaline solution to soften the aged horn, decompose the grease balance the skin secretion, leading out the impurity and metal ions at the same time. The electronic ions of positive electrode may lead the acidic solution (shrinking lotion) into the skin, and harden the skin tissue, comfort nerve and diminish blood supply as well. It is mainly used after beauty treatment to alleviate skin being red and swollen and to avoid inflammation. The electronic ions of negative electrode may lead alkaline solution into skin to soften skin tissue, irritate nerve and increase blood supply. Mainly use in aged and greasy skin.

2) Usage:

A: Client grasps the metal pole, and beautician holds the handle of the roller.

B: Use the "Galvanic" function selection button to adjust the introduction of nutrients. Then treated part must be clean and keep it clean and dry before doing the nutrition leading in. Turn the electrode button to (IN), now the roller is positive while the metal pole is negative. Beautician spreads some liquid nourishing lotion evenly on the treated part, then cover the roller with a piece of cotton or sponge that has dipped into the acidic lotion (like shrink lotion) or directly cover the roller with a piece of cotton with nourishing lotion before doing this treatment. While doing leading out treatment, should change the button to (OUT), now the roller is negative while the metal pole is positive. Cover the roller with piece of cotton or sponge that has dipped into alkaline lotion (like detergent lotion) before doing this treatment. It is to balance and decompose the grease of skin.

C: Adjust the button to the minimum, then hold the roller to the client's treated part, and slowly increase the energy position to the customer with a slight needle feel. Then gently slide the roller, the movement is slow, and the cotton sheet should be kept moist at all times to achieve the best results.

D: When working, whether the current loop is formed between the roller and the metal pole is detected. When there is no current loop for 5 seconds, the countdown automatically stops. When there is a current loop, the operation and countdown are resumed immediately.

4. Rotary Brush

The Rotary Brush is for the purpose of massaging. It can also help cleaning skin to remove the aging homy layer, impure substance and dead skin. As consequence, it can clean skin and improve skin quality, improve blood circulation, eliminate fatigue. The massaging effect can help skin become more elastic and brilliant.

There are large size brush head and small size brush head choosing. Beautician can choose it base on specific needs.

3. Product Technical parameters

Item	Specification
Input power	AC 220-240V 50/60Hz or AC 100-120V 50/60Hz
Maximum Input Power	60W
Package size	45×36×18.5cm
Net Weight	5.5Kg±5%
Gross Weight	6.5Kg±5%
Ambient Temperature	0°C ~ +45°C
Storage/Transport Temperature	-20°C ~ +60°C
Ambient Humidity	30%RH ~ 90%RH
Storage/Transport Humidity	30%RH ~ 80%RH

4. Packing List

NO	Name	Specification	Quantity
1	Host		1PC
2	Rotary Brush Handle		1 PC
3	Brush head set	large brush, small brush Large (small) grinding head	1 PC
4	High Frequency electrode set	Ladle-shape, Dot-like, Straight	2 PCS
5	Rubber Tube		1 PC
6	Glass Vacuum Pipes	Sharp-crested/ Flat mouth/ Round mouth	1 PC
7	Stick	Black	2 PCS
8	Spray Bottle	White	1 PC
9	High-frequency Handset	Black, Straight Line	1 PC
10	Input and Output Hand Stick	Black	1 PC
11	Power Cable	Black	1 PC
12	Introduction		1 PC
13	Gas Valve rubber seal	Black	2 PCS

The above parts are shown in Graph 1 below.

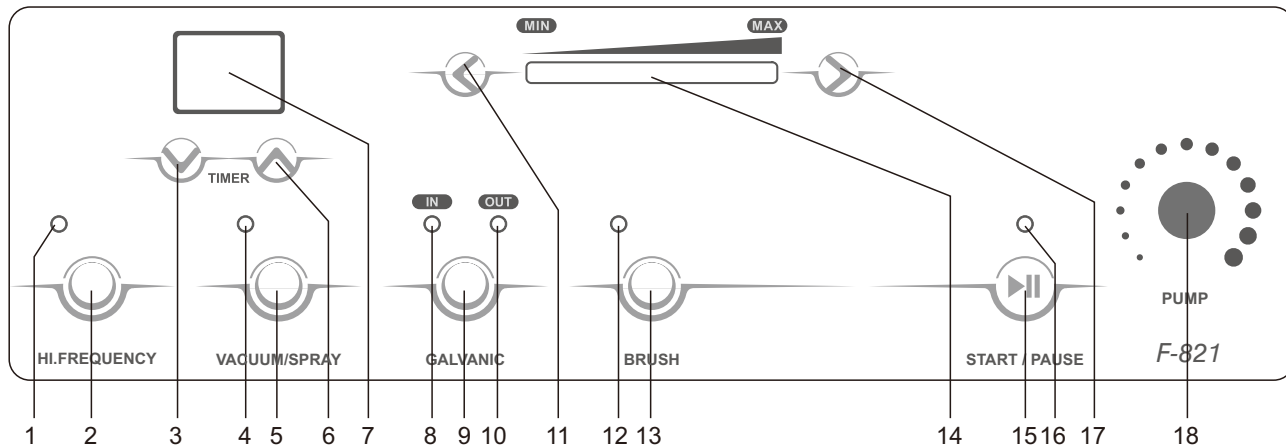


图1

5. Panel and Button Introduction

This instrument including five functions: High-frequency, Vacuum & Spray, Galvanic, Rotary brush. Buttons and indicators, display instructions are shown in Graph 2.

1.



1. High Frequency Electrotherapy Indicator Light
2. High Frequency Electrotherapy Function Selection Button
3. Time-
4. Vacuum & Spray Indicator Light
5. Vacuum & Spray Function Selection Button
6. Time+
7. Countdown Minute Display
8. Input Indicator Light
9. Galvanic Function Selection Button
10. Output Indicator Light
11. Energy-
12. Rotary Brush Indicator Light
13. Rotary Brush Function Selection Button
14. Energy Display
15. "START/PAUSE" Button
16. Work Indicator Light
17. Energy+
18. Suction Regulating Valve

1.High-frequency Function

- 1)In standby state, power on “HI.FREQUENCY” button, the indicator light on, then power on “START/PAUSE” button, high-frequency function start to work, the work indicator light on.
- 2)Time adjustment and energy adjustment can be performed in working state or standby state.
The setting range of time is:1-15 minutes, energy gear: 1-10.

2.Vacuum & Spray Function

- 1)In standby state, power on “VACUUM/SPRAY” button, the indicator light on, then power on “START/PAUSE” button, Vacuum & Spray Function start to work, the work indicator light on.
- 2)Time adjustment can be performed in working state or standby state. The setting range of time is: 1-10 minutes. The energy of the spray is adjusted by the nozzle on the spray bottle. The energy absorbed is the adjustment of the “Suction Regulating Valve”, the energy display at this time is independent of the “Energy+” button or “Energy-” button.

3.Galvanic Function

- 1)In standby state, power on “GALVANIC” button, the “IN” indicator light on,then press it again, will switch to “IN” indicator light on, then on “START/PAUSE” button, Galvanic Function start to work, the work indicator light on. When “Input Indicator Light” on, the roller is positive while the metal pole is negative; when “Output Indicator Light” on, the roller is negative while the metal pole is positive.
- 2)Time adjustment and energy adjustment can be performed in working state or standby state.
The setting range of time is: 1-15 minutes, energy gear:1-10.

4.Rotary Brush Function

- 1)In standby state, Power on “BRUSH” button, the indicator light on, then press “START/PAUSE” button. The Rotary Brush Function starts to work, the work indicator light on. Pressing “START/PAUSE” button again, the rotary function back to standby state.
- 2)Time adjustment and energy adjustment can be performed in working state or standby state.
The setting range of time is: 1-15 minutes, energy gear: 1-10.

PS: Will hear a buzzing voice each time press button, and a 1 second buzzing after finished counting backward, machine will back to standby state.

6 Attention

1.High-frequency-Cautions

- 1)The glass electrode pole will sparkle when it is still in less than 1 inch distance from the skin while the instrument turning on, please reduce the energy of Hi-frequency. If the client still feels the sparkle existing, may cover the face with small piece of dry cotton towel but not the nylon or synthetic fiber one.
- 2)For preventing electric shock, beautician must choose the minimum intensity before touching the client's face, and slowly increase the Hi-Frequency energy.
- 3)Before taking away the glass electrode pipe from the skin, must twist the “START/PAUSE” button to power off.
- 4)Under normal circumstance, the continuous working time of high-frequency electrotherapy should not exceed 30 minutes, otherwise it may cause the spare parts to overheat. At this time, it needs to be stopped for 10 minutes before use.

2.Vacuum & Spray-Cautions

- 1) Do not push spray bottle pipe into vacuum pipe marks nozzle prevent make-up water or other liquid being in-taken into air pump and damaging the instrument.
- 2) A filter sponge must be placed in the sticks to be used.
- 3) Do not work over 10 minutes, otherwise, it may cause the spare parts to overheat. At this time, it needs to be stopped for 10 minutes before use.

3. Galvanic-Cautions

- 1) When setting on "IN", the stung feeling will be felt more intensive, when setting on "OUT", it will be felt weaker.
- 2) Before doing the treatment, clients will be required to take off jewels, it is not applicable for someone who there are any metal objects inside.
- 3) The lotion with leading in or leading out must be used in accordance with the roller (negative or positive). While the roller is positive, the lotion should be acidic(+), while the roller is negative, the lotion should be alkaline(-). Please use the PH paper for verification if the lotion is not indicated.
- 4) The roller can't stop moving, otherwise stung feeling will be felt more intensive.

4. Brush- Cautions

- 1) Confirm if brush head is completely fixed before using.
- 2) Lower the brush speed before its head touch skin.
- 3) Long time treatment is forbidden, or it will produce heat on the handle.
- 4) Do not change brusher head while still working with one.
- 5) Do not use hand or other objects to change rotation direction.

7 Daily use and maintenance.

- 1) All kinds of glass electrode pipe should be kept clean, dry, disinfected before and after use.
- 2) When stop using Vacuum & Spray, turn on the switch "POWER", then turn counterclockwise the "Suction Regulating Valve" to the loosest, finally remove all air pipes and accessories for disinfection.
- 3) The filter sponge in the sticks needs to be replaced periodically according to the usage, and the inner pipe of suction rod should be cleaned if necessary.
- 4) When stop using Galvanic, the roller and metal pole must be clean and keep in dry to avoid damage.
- 5) The Brusher should be taken out, clean & disinfected after use. Put it back to accessory box and store it in dry place.
- 6) It will influence airtightness if the rubber seal in knob of gas valve wear. It should be replaced base on real use situation.

8 Possible problems of the system and maintenance methods

Fault phenomenon	Troubleshooting methods
Power on but no display	<ol style="list-style-type: none"> 1.Check whether the power cord is properly connected. 2.Check whether the power switch is on and whether the switch indicator is on. 3.Check whether the input voltage conforms to the rated specification of the products. Check whether the fuse is blown. If it is blown, please replace the fuse of the same model. The power of the instrument must be disconnected before replacement(must ask the professional to replace it)
High frequency without output	<ol style="list-style-type: none"> 1)Check if the high-frequency handset socket connection is correct. 2)Check if the power supply voltage is consistent with the nameplate of the appliance. 3)Replace other glass electrode pipe to test. Replace high-frequency handset to test.
The sticks have no suction effect	<ol style="list-style-type: none"> 1.Check if the power supply voltage is consistent with the nameplate of the appliance. 2.Listen and check if air pump is running. 3.Check if there is any foreign matter inside the sticks or in the air tube. 4.Check if the sticks are connected to the "VACUUM" connector, if the connection is good or the air pipe is damaged. 5.Check if connection points of the sticks are loose, if necessary, retighten.
Spray bottle without spray or spray is poor	<ol style="list-style-type: none"> 1.Listen and check if the air pump is running. 2.Check if the power supply voltage is consistent with the nameplate of the appliance. 3.Check if the spray bottle air line is connected the "SPARAY", if the connection is good, or the air pipe is damaged. 4.Check if there is any foreign matter in the nozzle inlet, nozzle, and air pipe. 5.Check if the hose inside the spray bottle is off. First loosen the small nut of the suction nozzle, re-rotate and adjust the gas metal pipe to restore the spray effect, and then tighten the small nut of the suction nozzle.
Galvanic has no output	<ol style="list-style-type: none"> 1.Check if the plug of the handset is connected correctly and check if the accessory cable is damaged. 2.Check if the metal pole and the roller act on the dame person, and the contact surface is well conductive.
The Brusher doesn't Rotate	<ol style="list-style-type: none"> 1.Check if it connected correctly to the output. 2.Check if brush cable is completely connected, or damaged. Check if the motor of brusher were stuck by something eles.
Other fault phenomenon	In the event of a malfunction that cannot be eliminated by yourself, or when a dangerous situation may occur, contact the dealer or manufacturer for further analysis and handling.

1001AT

Magnifying Lamp

User Manual



Welcome to use this product! A0
Please read this instruction carefully before use.

About This Manual


This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, we will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of manufacturer.

Safety Guidance

To ensure your correct and safe use of this product, please read all the instructions before use! Please follow all safety precautions!

	Warning!	<p>Indicates a potentially dangerous situation that, if not avoided, could result in serious loss of life.</p> <ul style="list-style-type: none">• This product is only for beauty practitioners with professional qualifications.• Do not use or store the product in a damp place such as bathroom to avoid malfunction and danger.• Do not use this product near flammable items (e.g., alcohol, thinner, gasoline, nail polish remover, sprays, etc.).• Do not use this product under the following conditions: when the power cord or plug is damaged or gets hot; when the plug in the power socket comes loose.• Do not use wet hands to plug and unplug power cord connectors and use this product.• Please make sure that the working voltage of this product conforms to its rated voltage.• Before use, make sure that the power plug and the power cord connector are fully plugged in, so as not to cause danger.• When not using this product, please turn off the power and pull out the plug.• Please regularly clean the dust accumulated on the power plug and power cord connector to avoid failure of insulation caused by moisture, dust and other factors.• Be sure to use the special power cord assembly for this product. Do not use the power cord assembly on any other appliance.• Do not wrap the power cord around the product, otherwise it may distort and malfunction the power cord.• Keep out of reach of children. Do not allow children to use this product.• Please do not modify, disassemble or repair this product by yourself. If you need to repair it, please contact the manufacturer, agent or professional maintenance personnel to check and repair it, so as not to cause danger.• Please contact distributor, service center or related personal with appropriate skill and experience for repairing if the plug is damaged, in case of any danger.
	Attention!	<p>Represents a potentially dangerous situation which, if not avoided, can result in general injury to person.</p> <ul style="list-style-type: none">• Operators should pay special attention to the safety of personnel and equipment when using.• If the product is abnormal, please stop using it immediately and contact the manufacturer or dealer for relevant treatment.• This product and related accessories should be handled lightly.• Please properly handle the packaging bag of this product to avoid accidents such as asphyxiation of infants, and keep it out of reach of children.

Cautions

- (1) Do not use in wet environments (like the bathroom). Clean your skin before operating.
- (2) Turn the power off after operation.
- (3) Please disconnect the product from power source if it is not being used for a long period of time. Clean the lamp head and cover it properly for storage.
- (4) Please wipe the lens with soft lens cleaning paper when cleaning the lens. Do not wipe with cloth and paper of other materials, otherwise the lens may be damaged.
- (5) Please avoid direct light projection to the eye.
- (6) If the external soft power cable is damaged, it can only be replaced by the manufacturer, service agent or personnel with similar qualification in order to avoid hazard.
- (7) WARNING “  (IEC 60417-6042): The light source can only be replaced by the manufacturer, service agent or personnel with similar qualification.
- (8) Do not use alcohol or other organic solvent to clean the lens.

Instruction

This device mainly detects changes in the human skin surface by examining the various minor changes in the skin surface. Providing further effectiveness when operating beauty and care treatments.

The magnifying function of the product can also be used for beauty treatments such as eyebrow and lips tattooing and eyelash grafting, etc.

The magnifying lens can be used independently with the supporting stand or in combination with other devices.

The magnifying lamp uses LED light source, energy saving and eye protecting, the brightness of the lamp is gentle and comfortable, users will not feel dizzy after using.

With innovative design and elegant profile, white glass with diopter of 5 is used to eliminate phenomenon of distortion, providing better visual effect.

Technical Specifications

- (1) Input power frequency and voltage as below:

Rated Voltage (V)	AC 100-240V
Rated Frequency (Hz)	50/60Hz

- (2) Rated power and illumination: (illumination tested at a distance of 20cm)

6.0W, 2500 LUX

- (3) Diopter of Len: 5D

- (4) Working environment: Temperature -20°C ~ +45°C, relative humidity 30%~90%

- (5) Storage environment: Temperature -20°C ~ +60°C, relative humidity 30%~80%

- (6) Net weight: 7.8kg


- (7) Gross weight: 9.1kg

Directions Of Use

- (1) Plug in the cold light lamp to a power source.
- (2) Turn the power switch to ON, aim the magnifying lens at the skin surface that needs to be examined or undergo beauty treatment.
- (3) Adjust the distance between the lamp and skin to achieve the clearest status
(the optimum visual range: $\leq 10\text{cm}$)
- (4) Examine the skin or begin beauty treatment. Brightness can be adjusted during operation.

Magnifying Lens

For combination

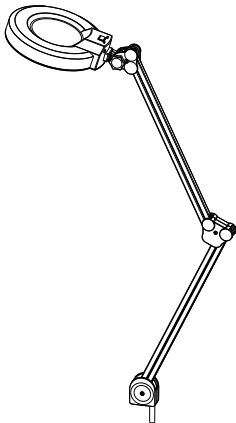

Do not dispose of old or defective appliances in domestic garbage; this should only be done through public collection points.

▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.

1004

Wood's Lamp

User Manual



Welcome to use this product! A0
Please read this instruction carefully before use.

About This Manual

This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, we will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of manufacturer.

Safety Guidance

To ensure your correct and safe use of this product, please read all the instructions before use! Please follow all safety precautions!

	Warning!	<p>Indicates a potentially dangerous situation that, if not avoided, could result in serious loss of life.</p> <ul style="list-style-type: none">• This product is only for beauty practitioners with professional qualifications.• Do not use or store the product in a damp place such as bathroom to avoid malfunction and danger.• Do not use this product near flammable items (e.g., alcohol, thinner, gasoline, nail polish remover, sprays, etc.).• Do not use this product under the following conditions: when the power cord or plug is damaged or gets hot; when the plug in the power socket comes loose.• Do not use wet hands to plug and unplug power cord connectors and use this product.• Please make sure that the working voltage of this product conforms to its rated voltage.• Before use, make sure that the power plug and the power cord connector are fully plugged in, so as not to cause danger.• When not using this product, please turn off the power and pull out the plug.• Please regularly clean the dust accumulated on the power plug and power cord connector to avoid failure of insulation caused by moisture, dust and other factors.• Be sure to use the special power cord assembly for this product. Do not use the power cord assembly on any other appliance.• Do not wrap the power cord around the product, otherwise it may distort and malfunction the power cord.• Keep out of reach of children. Do not allow children to use this product.• Please do not modify, disassemble or repair this product by yourself. If you need to repair it, please contact the manufacturer, agent or professional maintenance personnel to check and repair it, so as not to cause danger.• Please contact distributor, service center or related personal with appropriate skill and experience for repairing if the plug is damaged, in case of any danger.
	Attention!	<p>Represents a potentially dangerous situation which, if not avoided, can result in general injury to person.</p> <ul style="list-style-type: none">• Operators should pay special attention to the safety of personnel and equipment when using.• If the product is abnormal, please stop using it immediately and contact the manufacturer or dealer for relevant treatment.• This product and related accessories should be handled lightly.• Please properly handle the packaging bag of this product to avoid accidents such as asphyxiation of infants, and keep it out of reach of children.

Warning

1. Power supply must be cut off when changing electrical components or maintaining and cleaning machines.
2. The plug must be plugged into an outlet that is properly installed and grounded, to prevent the risk of fire and electric shock.
3. Do not allow children to play with and place the machine in a place where children can avoid touching.
4. Please do not place the machine in damp places such as bathroom to avoid electric shock or other failure.
5. Please do not place the machine in places where painting/coating or oxygen providing during operation.
6. Unless being monitored or instructed by personnel who can be responsible for their personal safety, the people who have physical obstacles, tactile disorders, mental disorders, lacking of experience, common sense or children, are not allow to use the machine.
7. Please contact distributor, service center or related personal with appropriate skill and experience for repairing if the plug is damaged, in case of any dangerous.
8. Maintenance should be processed by trained professionals.
9. This product can only be used by professional beauty practitioners, please pay close attention to working condition of the machine during operating.

Instruction

Ultraviolet ray lamp is made from general ultraviolet ray which go through the glass-filter with nickel. Wood's lamp was invented by American physical scientist Robert Williams Wood's. The main medical usage is to diagnose the unfavorable condition of skin, such as rash.

Usage

Beautician use wood's lamp to examine the condition of skin. Different material irradiated with dark purple ray of wood's lamp will appear ray with different colors. The wood's lamp is supposed to be used in a dark room. The purple ray can be used to examine the scarf skin of human body by beautician. What's more, the beautician can do some appropriate skin care for the customer. The invisible skin blemish can be displayed by wood lamp. The displayed shape and color of different skin would be different. For example, if the skin is too thick, the displayed ray should be bright white. The following cases are some of the situation under wood's lamp.

Different type of skin under wood's lamp appears different color.

Skin condition	Displayed color
Thick epidermis	White fluorescence
Necrosis cells	White spot
Healthy skin	Blue and white
Water deficiency(thin skin)	Purple
Water deficiency	Light purple
Water abundance	Bright fluorescence
Dark fleck	Brown
Oiled part and pimple	Yellow or pink

The healthy skin on head and hair under wood lamp should appear white and grey purple. The nail should appear bright white.

If you doubt that the formula of cosmetic changes, you can put the previous and current cosmetic under the wood lamp for checking. The same formula of cosmetic should appear the same color.

Mostly the wood's lamp are applied on face of human. First, we hook up the sunshade cover on wood's lamp, and then hook up the other end of sunshade cover on customer's face. The operation should be done in dark room.

Attention

1. While doing the examine, the customer must close their eyes or put cotton cloth on their eyes. Also the beautician are not allowed to stare at the lamp house.

2. The working time couldn't be more than 30 minutes; the tube mustn't be in touch of the skin directly, and shouldn't explore on the customer's skin for long time.



3. Should the powerline was damaged, to avoid danger, you should specify it to the manufacturer or the nominated maintenance department, taking apart and replacing privately is prohibited.

Technical parameters

Weights and sizes	Product size (cm)	30×21×7
	Net weight (kg)	0.9
	Gross weight (kg)	1.2
Electrical characteristics	Rated voltage	AC 100-120V
	Frecuency	50/60Hz
	Rated power	22 W



Do not dispose of old or defective appliances in domestic garbage; this should only be done through public collection points.

- ▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.